


ARJO


CONCERTO/BASIC

PARTS LIST

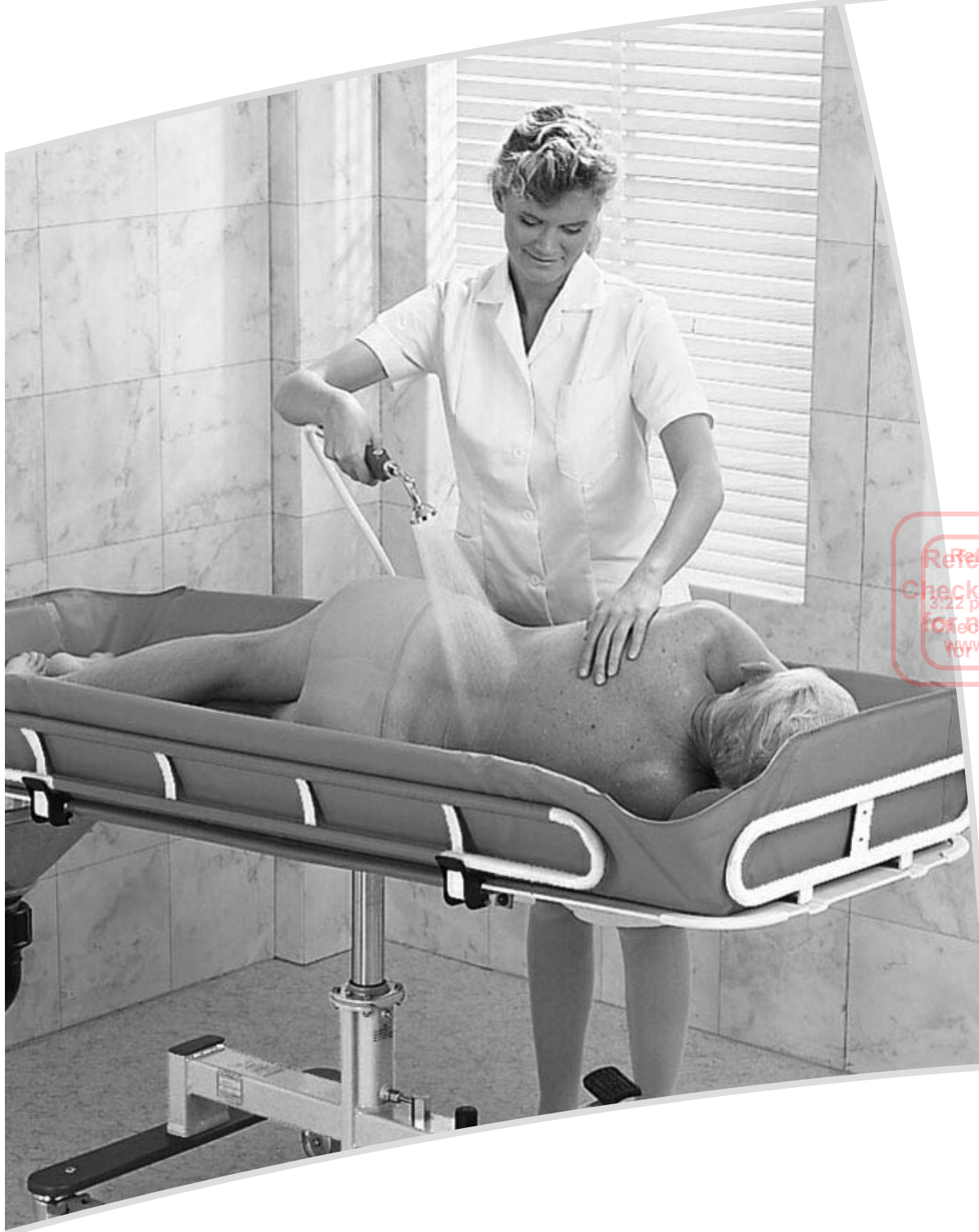

Reference Copy
Printed
3:22 pm, Jan 09, 2006
Check occasionally
for new editions
www.arjo.com

05.BA.00/3GB
January 2006

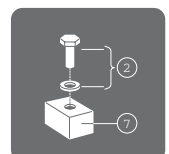



How to order

Before ordering spare parts study the exploded view closely and make sure that you note all involved spare parts.

Please note that gaskets and o-rings normally are included in kits and can not be ordered separately.

Note. all article numbers starting with **85** indicates that more than one detail is included in the article number.

When taking contact with ARJO Parts sales always have the following information at hand:

- Correct name of the product. See type label. The serial number of the product. See labels.

Manufacturing country
SE=Sweden

Manufacturing works
E=Eslöv

Year

Week

Serial number ←

SEE0102001

Explanation of the serial number:

Product no
AR31211-EU

Serial no
SEE0102001

Made by Arjo in Eslöv, Sweden
010115

Example of identification label

Old parts

This Parts List does not show complete history. For old parts not found please refer to previous Part List 5.08.04/3.

Modifications made to the equipment without the express acknowledgement of ARJO will invalidate supplier's product liability.



Overview	4
A1 Electric Chassis (Concerto)	6
A2 Electric Pump/Battery Box (Concerto).....	8
B1 Hydraulic Chassis	10
B2 Hydraulic Pump.....	12
C Stretcher	14
D Horizontal Adjustment (Concerto)/Back Support ...	18
E Battery Charger/Battery (Concerto)	20
Preventive Maintenance Parts/Recommended Spares	23


Overview

Pos	Description	See page
A1	Electric Chassis (Concerto)	6
A2	Electric Pump/Battery Box (Concerto)	8
B1	Hydraulic Chassis	10
B2	Hydraulic Pump	12
C	Stretcher	14
D	Horizontal Adjustment/Back Support (Concerto)	18
E	Battery Charger/Battery	20





Overview

Reference Copy
Printed
3:22 pm, Jan 09, 2006
for new editions
www.arielibrary.com

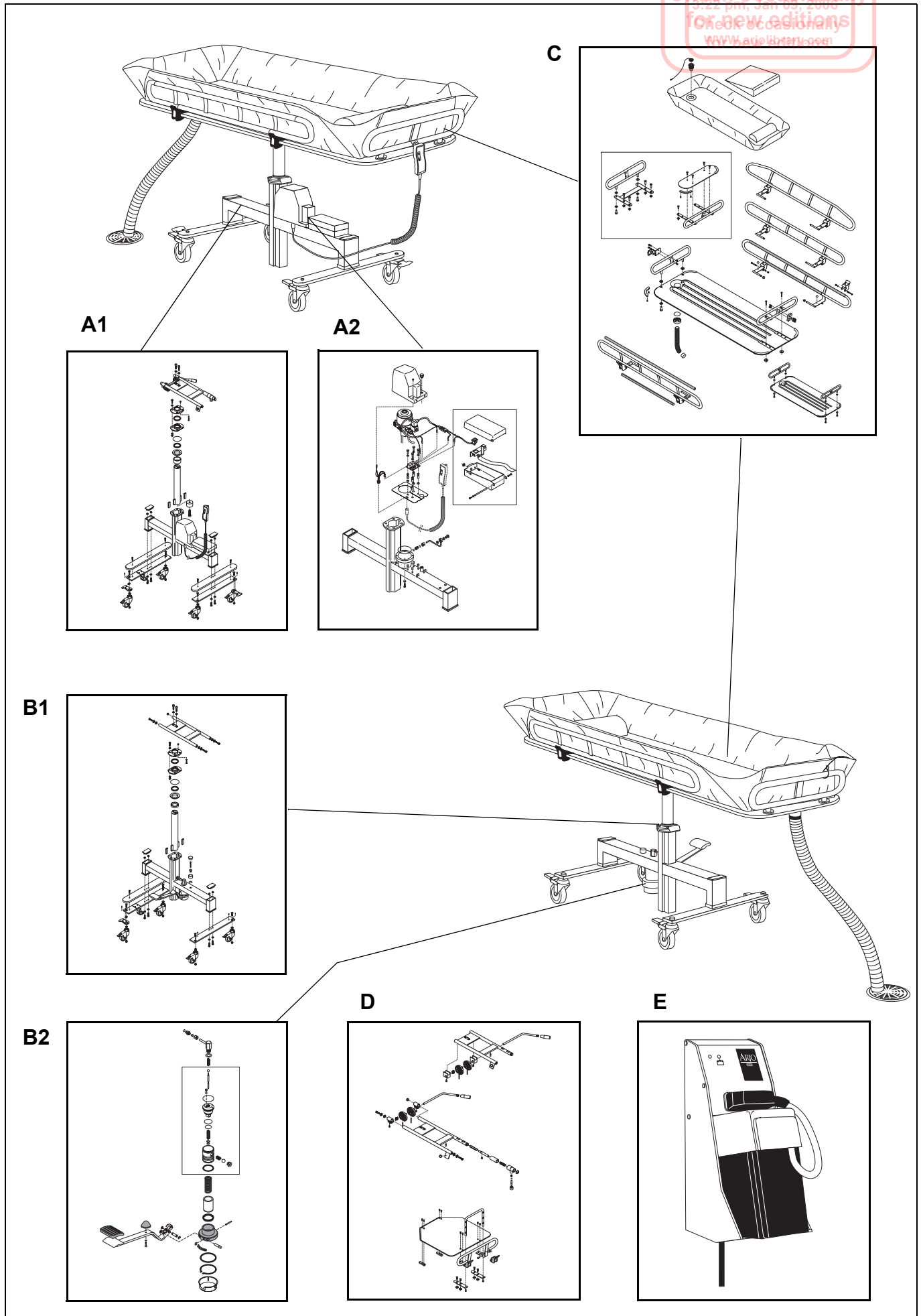


AI		Electric Chassis (Concerto)				Remarks
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	
1	6100630	4	Screw M6S 6x30 A4 DIN 931			
2	6200003	8	Washer BRB 6,4x12 A4 DIN 125			
3	6210023	1	Nut MHM M4 A4			
4	8421223-04 8421223-031	1	Washer, white Washer, grey			
5	6300002	1	Scraper			
6	8422062-013	1	Bearing, dark blue			
7	6261202	1	O-ring			
8	8435159	1	Nipple			
9	8536002-04 8536002-031	1	Bearing complete, white Bearing complete, grey			
10	6260065	1	O-ring			
11	6321005	1	Seal			
12	8422059	1	Guide			
13	8435771	1	Stop ring			
14	8322055 8343332	1	Lifting piston Lifting piston, 1600mm stretcher			
15	8321060-01	2	Guide piece			
16	6210043	4	Locking nut M6 A4			
17	8238118-04 8238118-031	1	Chassis, white Chassis, grey			
18	8321086-013	2	End plug, dark blue			
19	6210015	4	Nut M6M M10 A4 DIN 934			
20	8551413-01 8551413-013	2	Wheel balk complete, white Wheel balk complete, grey			
21	6190337	4	Screw MRT-TT 6x12 A4			
22	8123945-013	2	Cover, dark blue			
23	8351402-04 8351402-031	2	Balk, white Balk, grey	000701		
24	6200007	2	Washer BRB 10,5x20 A4 8.8			
25	6101040	4	Screw M6S 10x40 A4 DIN 931			
26	6200007	4	Washer BRB 10,5x20 A4 8.8			
27	6200019	2	Washer TBRB 10,5x28 A4			
28	6311031 8546848	4 2	Wheel (Rhombus) Wheel kit	000701	000701	
29	8546849	2	Wheel kit, straight steering	000701		
30	6210045	4	Locking nut M10 A4 8.8 DIN 985			
31	6190089	2	Screw MC6S 10x22 A4-100 Bufoloc			
32	8524024	2	Pin kit			
33	8451407	2	Washer			
34	6307076	1	Handset			
35	6130825	1	Screw MC6S 8x25 A4 DIN 912			1600 mm chassis
36	8443334	1	Spacer			1600 mm chassis
37	8354924-031	1	Hose holder, grey			

A1

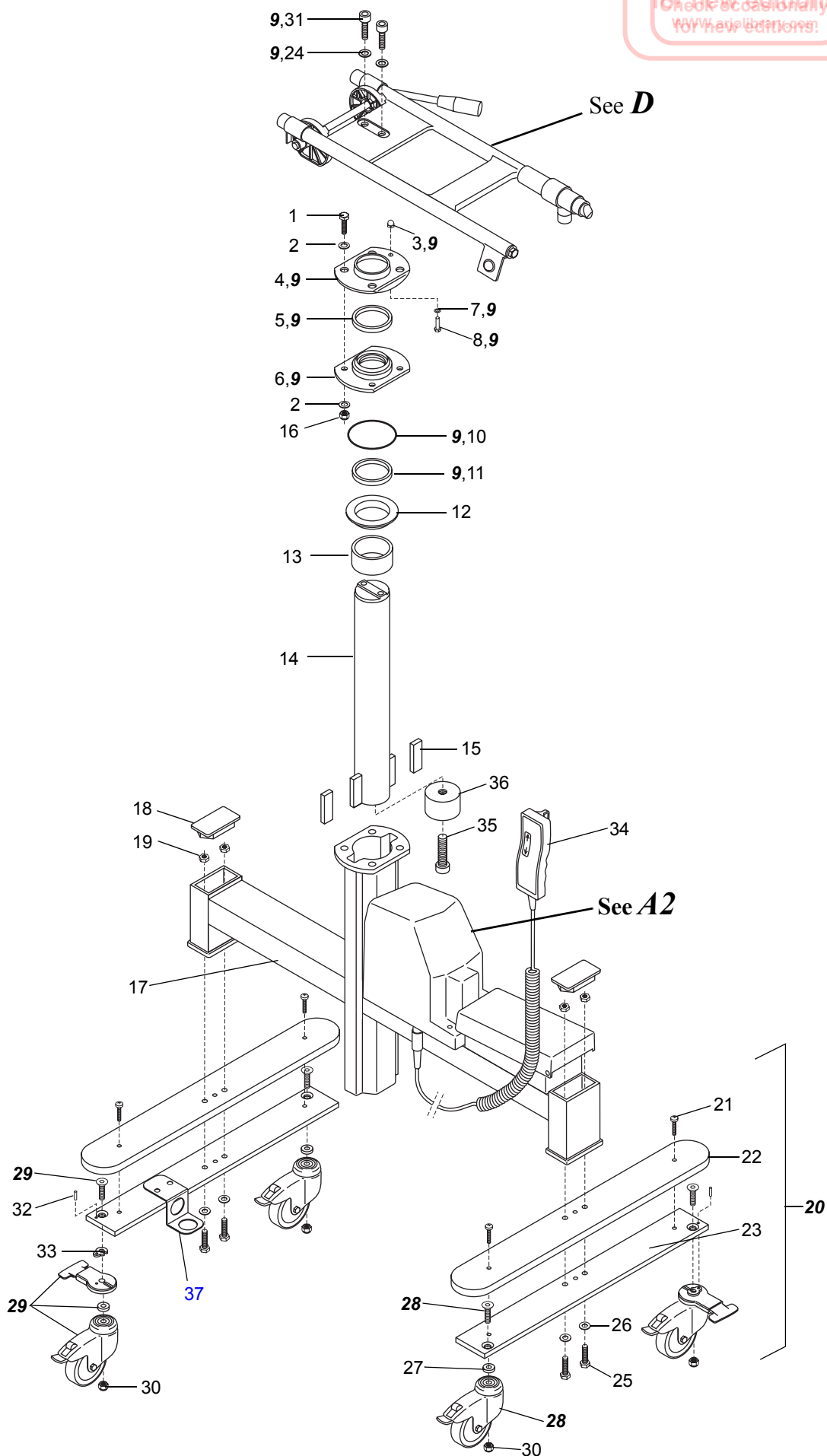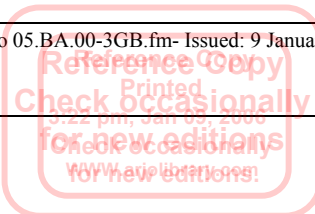

A2		Electric Pump/Battery Box(Concerto)				
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	6200003	12	Washer BRB 6,4x12 A4 DIN 125			
2	6100625	4	Screw M6S 6x25 A4 DIN 933			
3	6316076	1	Lid			
4	6301017	4	Buffer			
5	8332855	1	Protection plate			
6	8338119	1	Protection plate			
7	6210013	4	Nut M6M M6 A4 DIN 934			
8	6120316	4	Screw MCS 3x16 A4 DIN 84			
9	6130612	2	Screw MC6S 6X12 A4 DIN 912			
10	8435769	1	Cable microswitch	*		*New edition SEE0449195
11	6200082	4	Washer, 3,2x7x0,5 plastic			
12	8331275	1	PCB			
13	6303081	4	Spacer			
14	6907166	1	Pressure switch			
15	8439139	1	Cable solenoid valve	*		*New edition SEE0449195
16	6367020	1	LED-clip			
17	6316284	1	Hydraulic unit	SEE0449195		
18	6261207	1	O-ring			
19	8433404	1	Coupling			
20	6319163	1	Cutting ring			
21	6312032	1	Nut			
22	8333184 8457160	1	Tube	SEE0449195	SEE0449194	
23	6301008 8457155	1	Banjo screw	SEE0449195	SEE0449194	
24	6250293	2	Washer KB M14	SEE0449195		
25	6319485	1	Hose clamp	SEE0449195		
26	8133049-013	1	Cover, dark blue			
27	8546921	1	Replacement kit hydraulic unit		SEE0449194	Replaces 8233186
28	8454946	1	Cabling LED-handcontrol			
29	6270347	0,1m	Hose			
30	6270169-350	1	Hose			
31	6374022 8457034	1	Wire distributor	SEE0449195	SEE0449194	
32	6319445	1	Hose clamp			
33	6200152	4	Washer			
34	8356749	1	Harness, motor	SEE0449195		
35	8538116-04 8538116-013	1	Battery box white, complete Battery box grey, complete			
36	6370002	1	Rubber inlet			
37	8433181 8457189	1	Cable, batterybox	SEE0449195	SEE0449194	
38	6190108	2	Screw			
39	6200001	2	Washer			
40	8424964	1	Shaft			
41	8224236-013	1	Lid, dark blue			
42	6260065	1	O-ring			
43	6379077	1	Fuse T15AL 6,3x32mm			



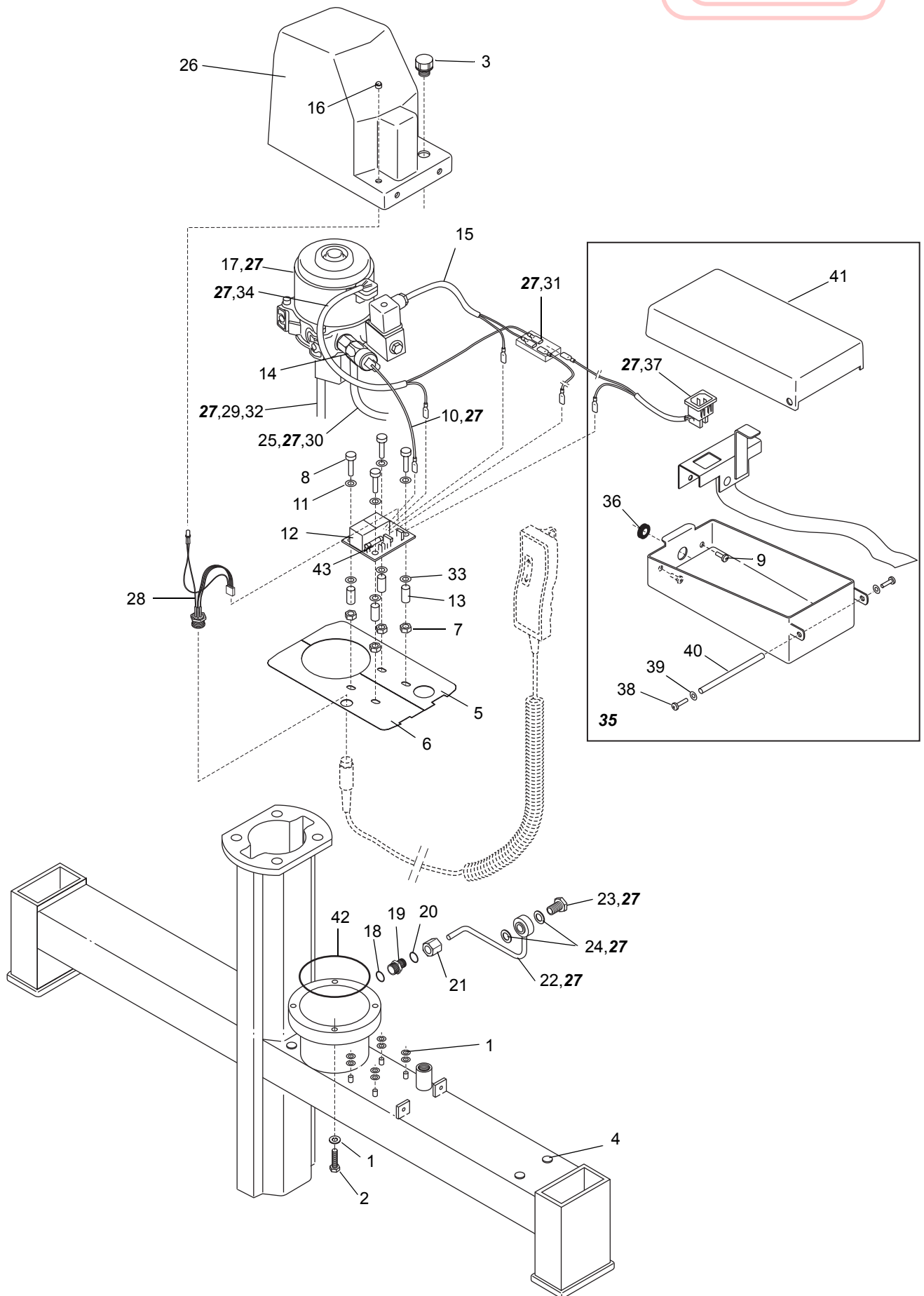

B1		Hydraulic Chassis				
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	6100630	4	Screw M6S 6x30 A4 DIN 125			
2	6200003	8	Washer BRB 6,4x12 A4 DIN 125			
3	8421223-04 8421223-031	1	Washer, white Washer, grey			
4	6300002	1	Scraper			
5	8422062-013	1	Bearing, dark blue			
6	6260065	1	O-ring			
7	6321005	1	Seal			
8	8422059	1	Guide			
9	8322055	1	Lifting piston			
10	8323502	2	Guide piece			
11	6210043	4	Locking nut M6 A4			
12	8422060	1	Stop ring			
13	8222107-04 8222107-031	1	Chassis, white Chassis, grey			
14	8321086-013	2	End plug, dark blue			
15	S8551413-01 S8551413-013 S8551410-04 S8551410-031	2	Wheel balk complete, straight steering Wheel balk complete, straight steering Wheel balk complete Wheel balk complete			Concerto, white Concerto, grey Basic,white Basic, grey
16	6190337	4	Screw MRT-TT 6x12 A4			Concerto
17	8123945-013	2	Cover, dark blue			Concerto
18	8351402-04 8351402-031	2	Balk, white Balk, grey	000701		
19	6200007	2	Washer BRB 10,5x20 A4 8.8			
20	6101040	4	Screw M6S 10x40 A4 DIN 931			
21	6200007	4	Washer BRB 10,5x20 A4 8.8			
22	6210015	4	Nut M6M M10 A4 DIN 934			
23	6200019	2/4	Washer TBRBSB 10,5x28 A4			Concerto 2, Basic 4
24	6311031 8546848	4 2/4	Wheel (Rhombus) Wheel kit	000701	000701	Concerto, Basic 4 Concerto 2, Basic 4
25	8546849	2	Wheel kit, straight steering	000701		Concerto
26	6210045	4	Locking nut M10 A4 8.8 DIN 985			
27	6210023	1	Nut MHM M4 A4			
28	6261202	1	O-ring			
29	8435159	1	Air nipple			
30	8542204	1	Oil plug, complete			
31	8411564	1	Screw			
32	6327001	1	End plug			
33	8423546	1	Oil plug			
34	6260047	1	O-ring			
35	8446002	1	Oil pin			
36	6190089	2	Screw MC6S 10x22 A4-100 Bufoloc			
37	6327086	2	Plug	000701		Basic
38	6327087	4	Plug	000701		Basic
39	8142210-04 8142210-031	1	Fixation plate, white Fixation plate, grey			Basic
40	8524024	2	Pin kit			Concerto
41	8451407	2	Washer			Concerto
42	6100820	3	Screw M6S 8x20 A4 DIN 933			Basic
43	8411342	3	Washer			Basic
44	8411343	3	Bush			Basic
45	6380013	1	End plug			Basic
46	8536002-04 8536002-031	1	Bearing complete, white Bearing complete, grey			
47	8354924-031	1	Hose holder, grey			Concerto

B1


Reference Copy
Printed
3:22 pm, Jan 09, 2006
Check occasionally
for new editions
www.eurolibraries.com

Concerto
See D

15 (Concerto)


See B2

15 (Basic)

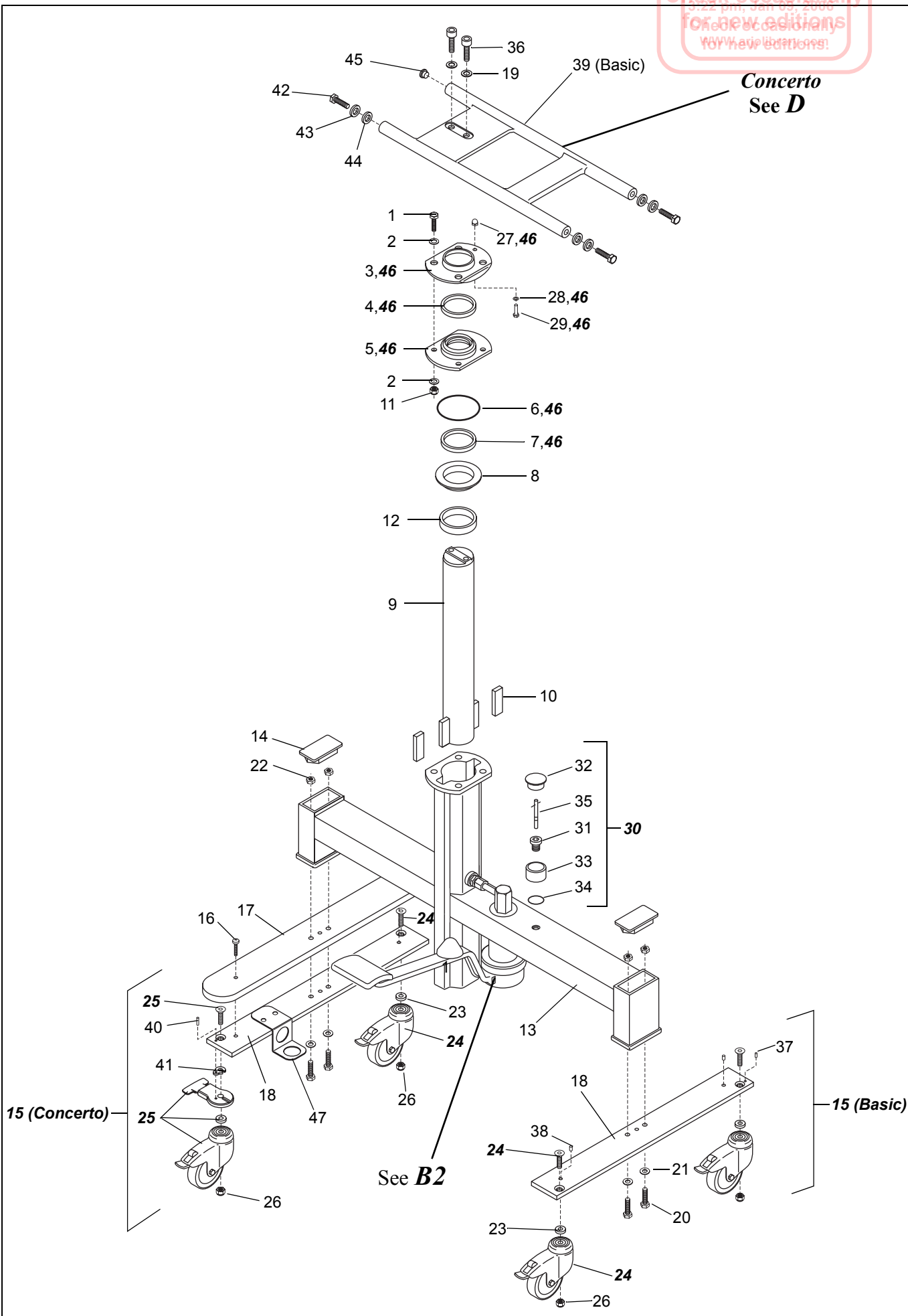

B2


Hydraulic Pump

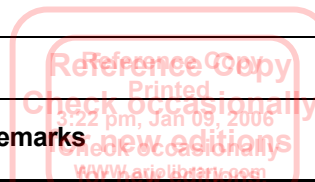

Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
A	8531802-04 8531802-031	1	Hydraulic pump white complete Hydraulic pump grey complete			Not incl. pos 1-6, 22-24, 32,33,35
B	8535483	1	Repair kit			
1	6260012	1	O-ring			
2	8420709	1	Straight coupling			
3	8421261	1	Cone			
4	8420712	1	Coupling			
5	6260041	1	O-ring			
6	8421260	1	Spring			
7	6310031	2	Ball			
8	8531174	1	Pump insert kit			
9	8421762	1	Lowering pin			
10	8420724	1	Plug			
11	6310028	1	Ball			
12	6260057	2	O-ring			
13	8321763	1	Pump body			
14	6260054	1	O-ring			
15	6260050	1	O-ring			
16	8420749	1	Spring			
17	6200001	1	Washer BRB 4,3x8 A4 DIN 125			
18	6319006	1	Circlip			
19	8421248	1	Suction nipple			
20	8421249	1	Spring			
21	8420907	1	Spacer			
22	8420913	1	Pressure spring			
23	8420904	1	Piston			
24	6310074	1	Seal			
25	8421764	1	Stopper			
26	8320725	1	Bearing ring			
27	6240330	1	Pin			
28	8420731	1	Pedal shaft			
29	6319032	3	Circlip			
30	8420732	1	Shaft			
31	8420733	1	Sleeve			
32	8421055	1	Sliding ring			
33	6319023	1	Circlip			
34	8221820-04 8221820-031	1	Pedal, white Pedal, grey			
35	8320746	1	Pump cover			
36	8321832	1	Pedal rubber			
37	6200003	1	Washer BRB 6,4x12 A4 DIN 125			
38	6100620	1	Screw M6S 6x20 A4 DIN 933			
39	8443822	1	Damper			

B2


Reference Copy
Printed
3:22 pm, Jan 09, 2006
Check occasionally
for new editions!
www.eurobar.com

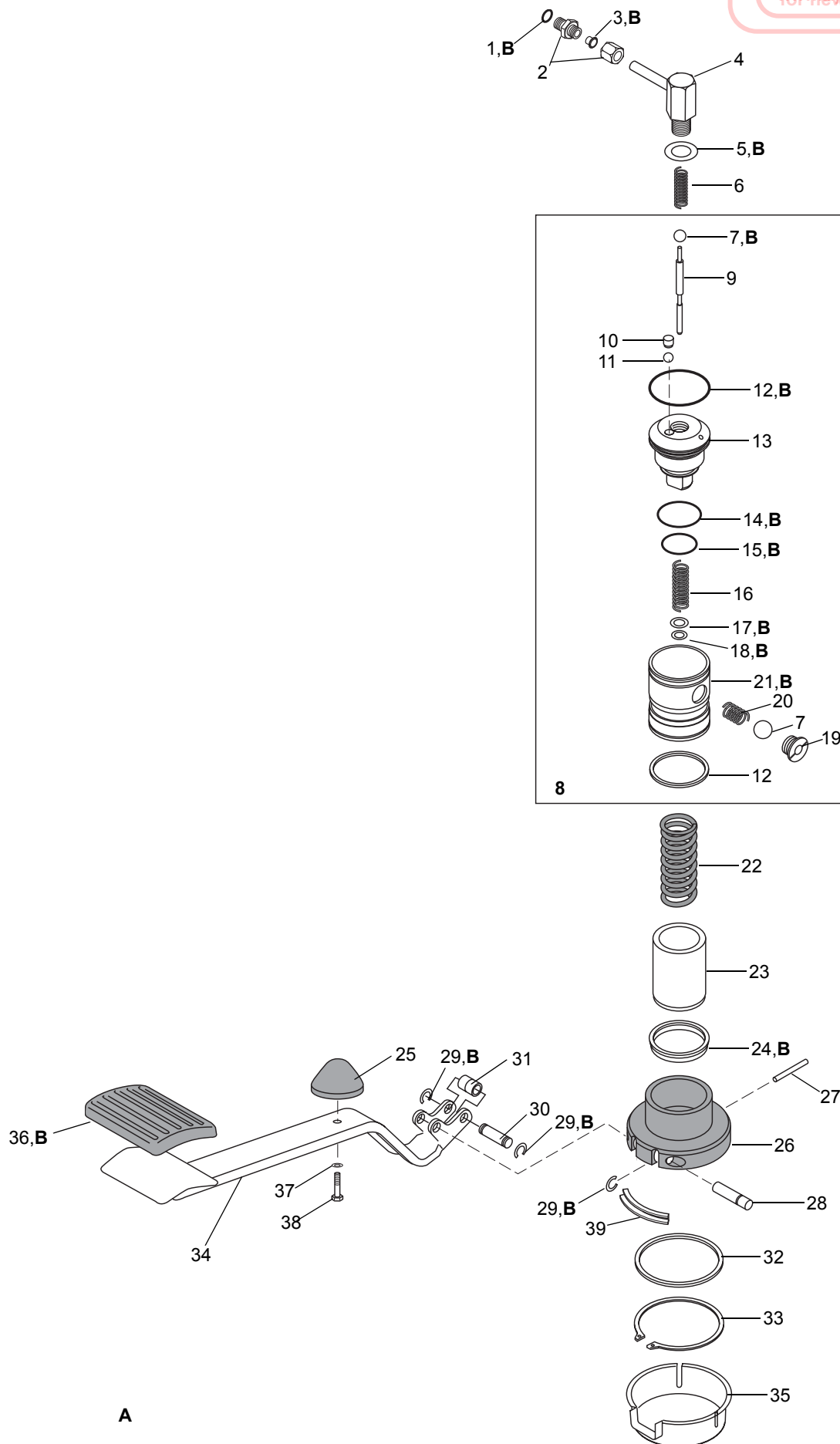

C <i>Stretcher</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8231551-013	1	Pillow, dark blue			
2	8131431-014 8131422-014 8131421-014	1	Mattress, light blue 1600 mm Mattress, light blue 1900 mm Mattress, light blue 2250 mm			
3	100828 100828-1500 100828-2000 100828-2500	1	Drainage hose, complete			1000mm 1500mm 2000mm 2500mm
4	8523863-014	1	Drainage plug light blue complete			
5	100880-01	1	Wedge pillow, light blue			
6	8457925-04 8457925-031	1	Holder handset, white Holder handset, grey			Concerto, electric chassis
7	6190426	2/4	Screw K6S 6x16 A4			Concerto, electric chassis 4 pcs
8	6200003	2/4	Washer BRB 6,4x12 A4 DIN 125			Concerto, electric chassis 4 pcs
9	6210028	2/4	Nut MHM M6 A4 DIN 1587			Concerto, electric chassis 4 pcs
10	8322068-04 8322068-031	1	Extension, white Extension, grey			2250mm stretcher
11	6200005	2	Washer BRB 8,4x17 A4 DIN 125			2250mm stretcher
12	6100820	2	Screw M6S 8x20 A4 DIN 933			2250mm stretcher
13	8322077-04 8322077-031	1	Plate, white Plate, grey			2250mm stretcher
14	6150612	4	Screw MF6S 6x12 A4 DIN 7991			2250mm stretcher
15	8208700-04 8208700-031	1/2	End support, white End support, grey			1: 1900mm/Basic, 2250mm stretcher 2: 1600mm stretcher
16	8425485	2/4	Washer white			2:1900mm/Basic, 2250mm stretcher 4:1600mm stretcher
17	8408696	2/4	Washer white, countersunk			2:1900mm/Basic 4:1600mm stretcher
18	6150825	2/4	Screw MF6S 8x25 A4			2:1900mm/Basic, 4:1600mm
19	8211963-04 8211963-031	1	Head support, white Head support, grey			1900mm stretcher
20	6150820	2/8	Screw MF6S 8x20 A4 DIN 7991			2:1900mm/Basic 8: 2250mm stretcher
21	8411097	2/8	Washer			2:1900mm/Basic 8: 2250mm stretcher
22	8222076-04 8222076-031	1	Head support, white Head support, grey			2250mm stretcher
23	8420030	1	Plug			
24	6262414	1	O-ring			
25	8420029	1	Sleeve			
26	6270101	1m	Hose			
27	8122980-04 8122980-031	1	Side support white, right Side support grey, right			1600 mm stretcher
28	8122978-04 8122978-031	1	Side support white, left Side support grey, left			1600 mm stretcher
29	8111318-04 8111318-031	2	Side support white Side support grey			1900mm/Basic stretcher
30	8122088-04 8122088-031	2	Side support white, high Side support grey, high			1900mm/Basic stretcher
31	8432063-031	2	Cover grey long			
32	8431416-031	2	Cover grey short			Concerto
33	6100550	4	Screw M6S 5x50 A4 DIN 931			
34	6200002	4	Washer BRB 5,3x10 A4 DIN 125			

C

Reference Copy
Printed
3:22 pm, Jan 09, 2006
Check occasionally
for new editions!
www.eurolibraries.org

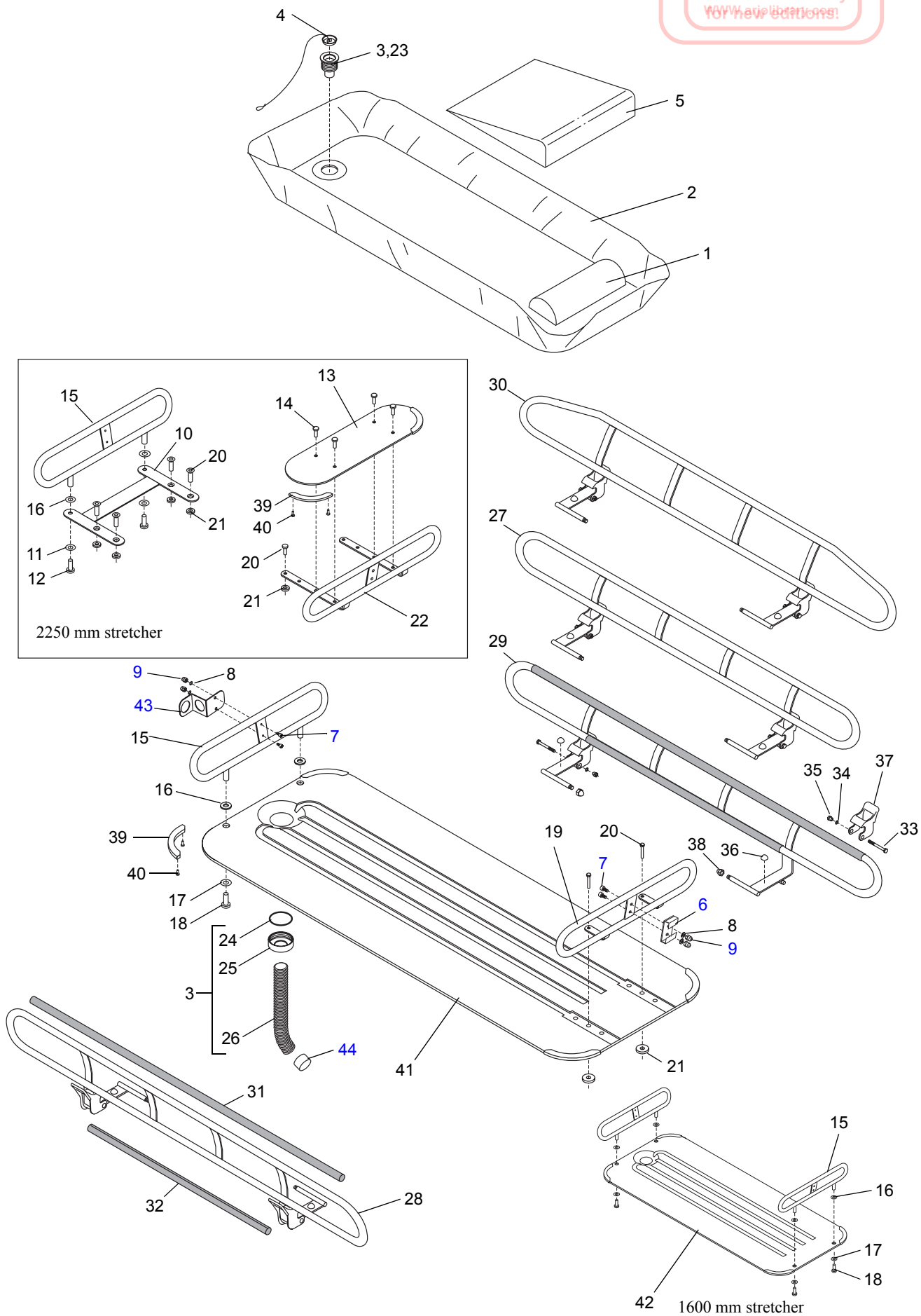


C <i>Stretcher</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
35	8425558	4	Nut			
36	6301018	4	Buffer			
37	8451622 8546608	4 1	Catch, complete Catch kit			Incl. 4 catches
38	6210024	4	Cap nut MHM M8 A4 DIN 1587			
39	8411517	4	End protection			
40	6314022	8	Rivet 3,2x9,9			
41	S8311978	1	Stretcher 1900 mm, complete			Concerto/Basic
42	S8323871	1	Stretcher 1600 mm, complete			Concerto
43	8357924-031	1/2	Hose holder, grey			Concerto 2 pcs/ Basic 1pcs
44	8408987	1	Tub			



C



Reference Copy
Printed
3:22 pm, Jan 09, 2006
Check occasionally
for new editions!
www.aai-libraries!

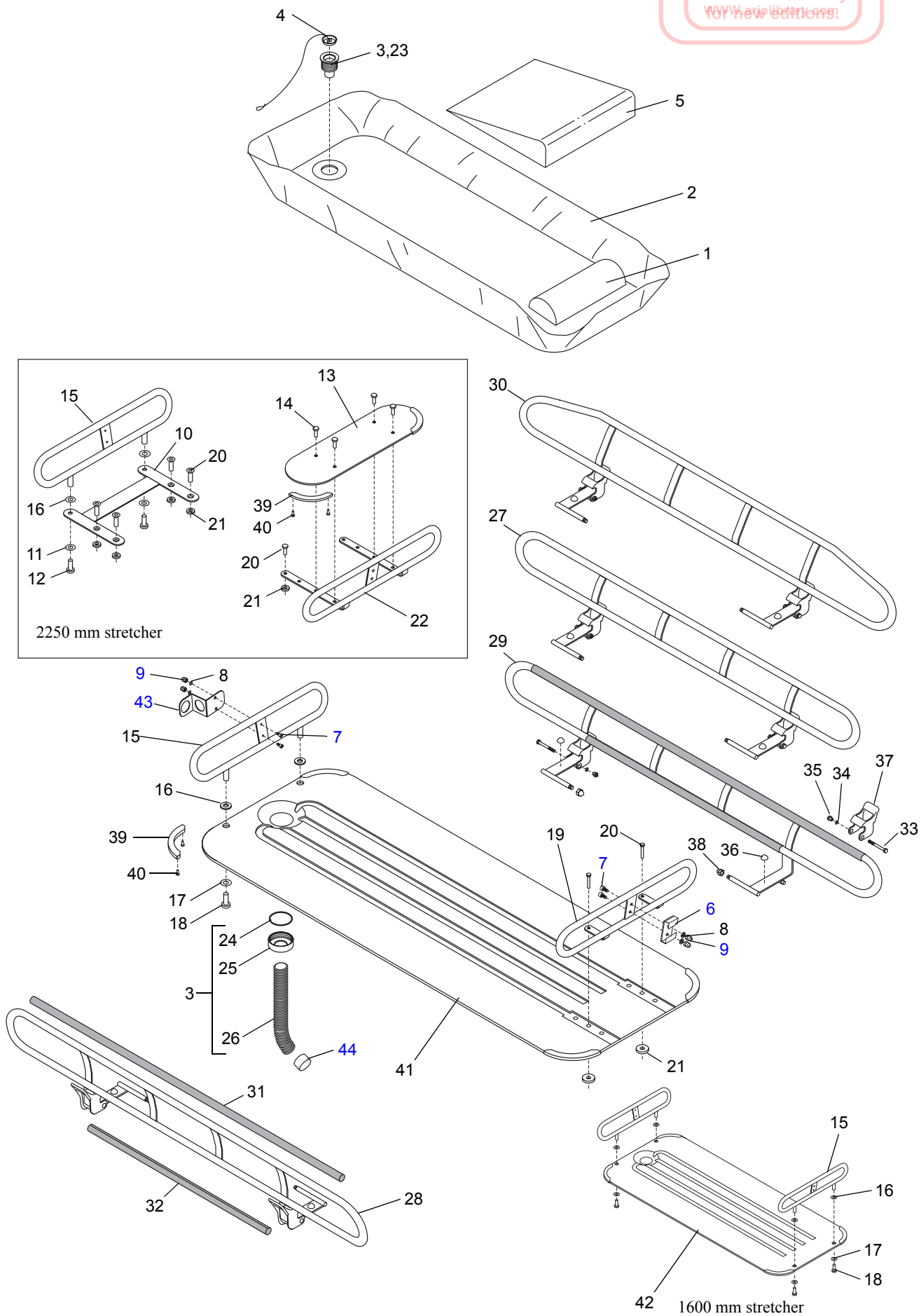

D <i>Horizontal Adjustment (Concerto)/ Back Support</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
A	S8533569-04		Fixation plate complete, white			1900/2250 mm
	S8533569-031		Fixation plate complete, grey			1900/2250 mm
	S8533573-04		Fixation plate complete, white			1600 mm
	S8533573-031		Fixation plate complete, grey			1600 mm
1	100804 S8523856	1	Horizontal adjustment lever complete			1900/2250 mm 1600 mm
2	8352683	2	Mounting block			1900/2250 mm
3	8423139	1	Mounting block for spring			1600 mm
4	8454792	1	Mounting block			1600 mm
5	6240450	2	Cylindric pin			
6	6130825	2	Screw MC6S 8x25 A4 DIN 912			1600 mm
7	6190087	2	Stop screw MSK6SS 8x16 A4			1900/2250 mm
8	8352659	2	Excenter			
9	8321770	1	Lever			
10	8421810	1	Spring			
11	6307005	1	Handle			
12	8122976-04 8122976-031	1	Fixation plate, white 1600mm Fixation plate, grey 1600mm			
13	8123949-04 8123949-031	1	Fixation plate, white 1900/2250mm Fixation plate, grey 1900/2250mm			
14	6220320	1	Resilient pin			
15	8424125	1	Pressure spring			
16	8423966	1	Spacer			
17	8423972	1	Screw			
18	8423973	1	Pressure spring			
19	8324454	1	Sleeve			
20	8423967	1	Nipple			
21	8423971	1	Catch pin			
22	6319085	1	Circlip			
23	8423974	1	Pressure spring			
24	8408674	1	Locking pin			
25	6319026	1	Circlip			
26	6327027	1	End plug			
27	6301018	1	Buffer			
28	6100820	2	Screw M6S 8x20 A4 DIN 933			
29	8411343	2	Bush			
30	8411342	2	Washer			
31	100806-04 100806-03	1	Adjustable back support, white Adjustable back support, grey			Complete
32	8323127	1	Clamp			
33	8223136-04 8223136-031	1	Handle, white Handle, grey			
34	6150820	4	Screw MF6S 8x20 A4 DIN 7991			
35	8423133-04 8423133-031	2	Fixation plate, white Fixation plate, grey			
36	6150816	2	Screw MF6S 8x16 A4			
37	6150610	4	Screw MF6S 6x10 A4			
38	8223134-04 8223134-031	1	Back support, white Back support, grey			
39	6150616	4	Screw MF6S 6x16 A4 DIN 7991			
40	8423128	2	Sliding block			
41	8411097	4	Washer			
42	8451622	2	Catch, complete			With spring and bumper
43	8408735	4	Screw			

Reference Copy
Printed
3:22 pm, Jan 09, 2006
Check occasionally
for new editions!
www.leonardo.it



D

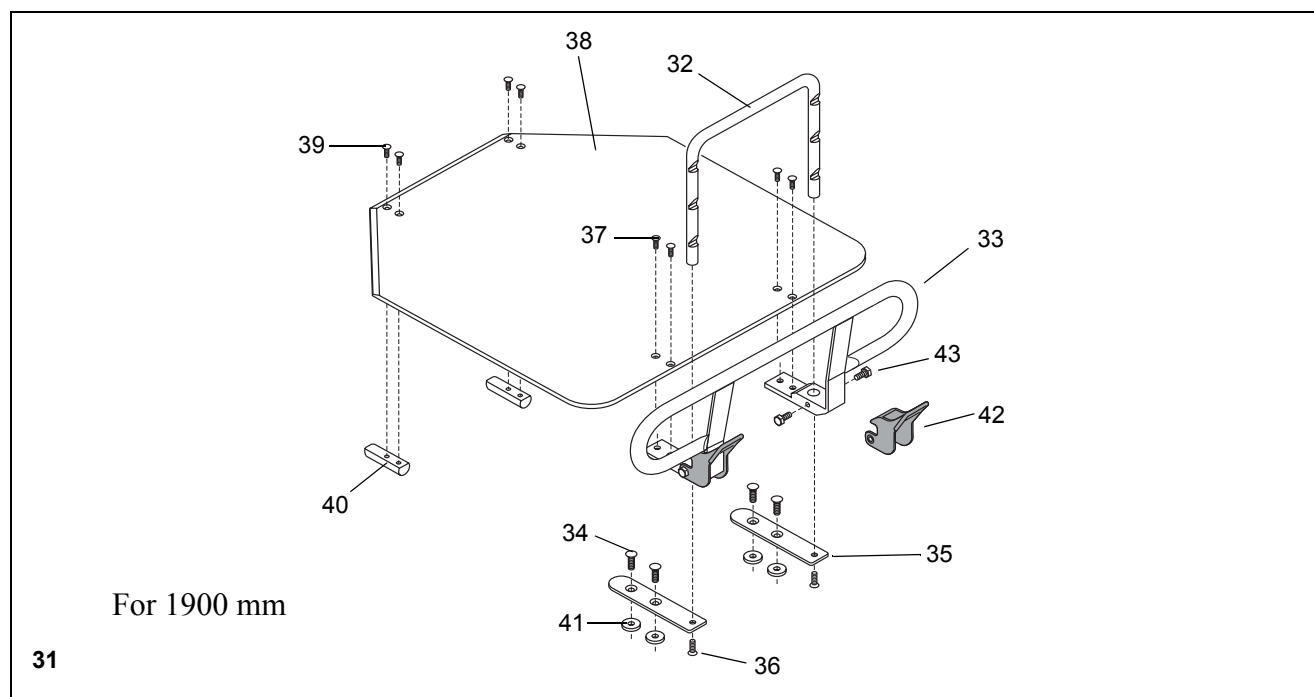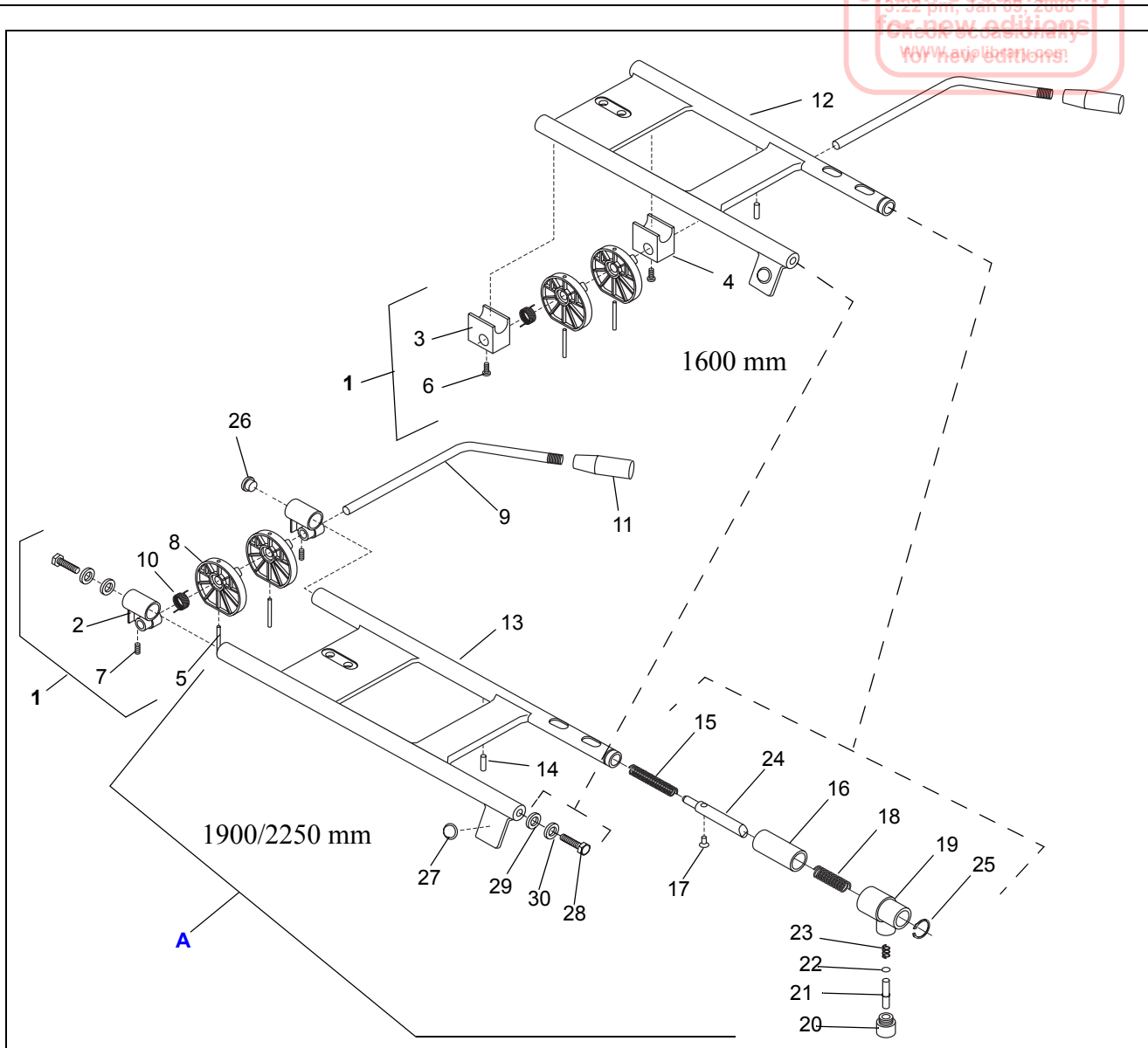

<i>E</i> <i>Battery Charger/Battery (Concerto)</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	NCA1000-EU	1	Battery charger			
	NCA1000-GB					
	NCA1000-CH					
	NCA1000-US					
	NCA1000-AU					
2	102193	1	Battery			



E

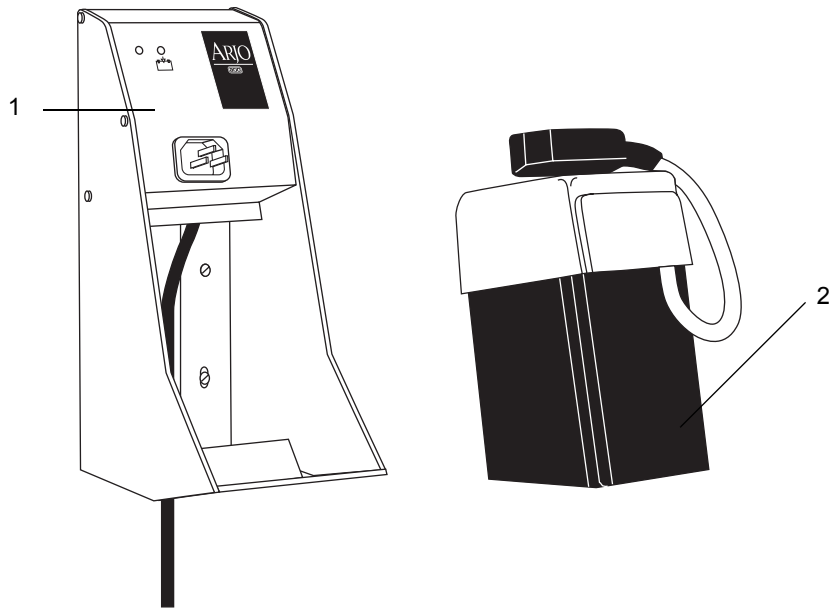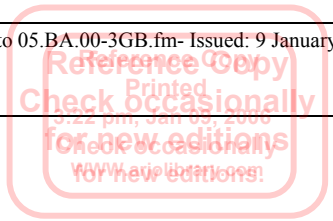

Preventive Maintenance Parts/Recommended Spares

Basic



Preventive Maintenance Parts	Part number	Number of Basic		Page
		1	5	
Catch kit	8546608	1	5	C:37
Wheel kit	8546848	4	20	B1:24
Old wheel up to 2000-07-01 (Rhombus)	6311031	4	20	B1:24

Recommended Spares	Part number	Number of Basic		Page
		1	5	
Mattress, light blue	8131422-014	1	2	C:2
Pillow, dark blue	8231551-013	1	2	C:1
Drainage plug light blue, complete	8523863-014	1	1	C:4
Catch with spring and bumper	8451622	2	4	D:42
Pedal rubber	8321832	1	1	B2:36
Pump cover	8320746	1	1	B2:35
Pump insert kit	8531174	1	1	B2:8
End protection	8411517	4	8	C:39
Bearing complete, white	8536002-04	1	1	B1:46
Bearing complete, grey	8536002-31			

Preventive Maintenance Parts/Recommended Spares Concerto



Preventive Maintenance Parts	Part number	Number of Concerto		Page
		1	5	
Catch kit	8546608	1	5	C:37
Wheel kit straight steering	8546849	2	10	A1:29, B1:25
Wheel kit	8546848	2	10	A1:28, B1:24
Old wheel up to 2000-07-01(Rhombus)	6311031	4	20	A1:28, B1:24

Recommended Spares	Part number	Number of Concerto		Page
		1	5	
Mattress, light blue 1600 mm Mattress, light blue 1900 mm Mattress, light blue 2250 mm	8131431-014 8131422-014 8131421-014	1	2	C:2
Pillow, dark blue	8231551-013	1	2	C:1
Wedge pillow, light blue	100880-01	1	1	C:5
Drainage plug light blue, complete	8523863-014	1	2	C:4
Catch with spring and bumper	8451622	2	4	D:42
Bearing complete, white Bearing complete, grey	8536002-04 8536002-31	1	1	A1:9, B1:46
Cover, dark blue	8123945-013	2	4	A1:22, B1:17
End protection	8411517	4	8	C:39
Hydraulic pump				
Pedal rubber	8321832	1	1	B2:36
Pump cover	8320746	1	1	B2:35
Pump insert kit	8531174	1	1	B2:8
Electric pump				
PCB	8331275	1	1	A2:12
Handset	6307076	1	2	A1:34
Battery	102193	1	2	E:2
Fuse T15AL 6,3x32mm	6379077	1	2	A2:43

AUSTRALIA

Arjo Hospital Equipment Pty Ltd
154 Lytton Road
BULIMBA
Brisbane QLD 4171
Australien
Tel: 07-3395 6311
Faxnr. 07-3395 6712

AUSTRIA

Arjo-Sic GmbH
Föhrenweg 5
6065 THAUR
Tel: 05223-49 33 50
Faxnr. 05223-49 33 50 - 75

BELGIUM

Arjo Hospital Equipment nv-sa
Ternesseelei 248
B-2160 WOMMELGEM
Tel: 03 353 91 00
Fax: 03 353 91 01
E-mail: info@arjo.be

CANADA

Arjo Canada Inc.
1575 South Gateway Road
Unit "C"
MISSISSAUGA, ON, L4W 5J1
Tel: 1-800-665-4831
Fax: 1-800-309-1116
E-mail: info@arjo.ca

CZECH REPUBLIC

Arjo Hospital Equipment s.r.o.
Strmà 35
616 00 BRNO
Tel.: 549 254 252
Fax: 541 213 550

DENMARK

Arjo Scandinavia
Postboks 51
4632 BJAEVERSKOV
Tel: 45 93 27 37
Fax: 45 93 27 39

FAR EAST

Arjo Far East Limited
1001-03 APEC Plaza
49 Hoi Yuen Road, Kwun Tong, Kowloon
HONG KONG
Tel: 2508 9553
Fax: 2508 1416

FINLAND

OY Vestek AB
Martinkuja 4
02270 ESPOO
Tel: 9 8870120
Fax: 9 88701291

FRANCE

Arjo Equipements Hospitaliers S.A.
45, Avenue de l'Europe
Eurocit BP 133
59436 RONCQ CEDEX
Tel: 03 20 28 13 13
Fax: 0 3 20 28 13 14
E-mail: info@arjo.fr

GERMANY

ARJO GmbH
Christof - Ruthof - Weg 6
D-55252 MAINZ-KASTEL
Tel: 06134-186-0
Fax: 06134 186 160
E-mail: info@arjo.de

GREECE

C. Psimitis Co Ltd
59, Dimitrion Str.
16121 KASARIANI, ATTIKIS
Tel: 210 724 36 68
Fax: 210 721 55 53

ITALY

Arjo Italia S.p.A.
Via Tor Vergata 432
I-00133 ROMA
Tel: 06-6566356
Fax: 06-65 66 32 12
E-mail: promo@arjo.it

THE NETHERLANDS

Arjo Nederland B.V.
De Blomboogerd 8
4003 BX TIEL
Postbus 6116
4000 HC TIEL
Tel: 0344-64 08 00
Fax: 0344-64 08 85
E-mail: info@arjo.nl

NORWAY

Arjo Scandinavia
Enebakkveien 117
N-0680 OSLO
Tel: 98 28 11 70
Fax: 22 57 06 52

POLAND

ARJO Poland Sp. z o.o.
Ul. Lirowa 27
PL-02-387 WARSZAWA
Tel: 22 882-06-26/28
Fax: 22 882-06-29

PORTUGAL

RTL
Edifício d'AS
R. Moreiró, 65 Gandra
4580 PAREDES
Tel: 224 119 070
Fax: 224 119 079

SPAIN

Arjo Spain, S.A.
C/ San Rafael, No. 6
28108 ALCOBENDAS
Tel: 091 490 06 36
Fax: 091 490 06 37
E-mail: arjospain@eresmas.net

SWEDEN

Arjo Scandinavia AB
Verkstadsvägen 5
Box 61
241 21 ESLÖV
Tel: 0413-645 00
Fax: 0413-645 83
E-mail: kundservice@arjo.se

SWITZERLAND


Arjo-Sic AG
Florenzstr. 1d
Postfach
CH-4023 BASEL
Tel: 061-337 97 77
Fax: 061-311 97 42

UNITED KINGDOM

Arjo Ltd
St Catherine Street
GLOUCESTER GL1 2SL
Tel: 08702 430 430
Fax: 01452-525 207

USA

Arjo Inc.
50 North Gary Avenue
ROSELLE, IL 60172
Tel: 1-800-323-1245
Fax: 1-888-594-2756

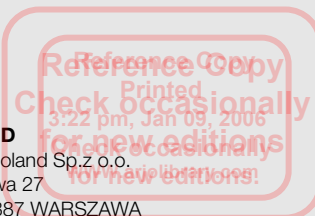

ARJO

...with people in mind.

www.arjo.com

info@arjo.com

MEMBER OF THE GETINGE GROUP



If your country is not listed here, please contact your local distributor or:
ARJO International AG, Florenzstrasse 1 d, Postfach-PO-Box, CH-4023 BASEL, SWITZERLAND,
Tel. +41 61 337 97 97, Fax: +41 61 331 47 80.