


ARJO

MAXI TWIN PARTS LIST



05.KT.00/3GB
August 2009

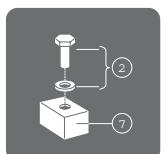

General information

How to order

Before ordering spare parts study the exploded view closely and make sure that you note all involved spare parts.

Please note that gaskets and o-rings normally are included in kits and can not be ordered separately.

Note, all article numbers starting with **85** indicates that more than one detail is included in the article number.

When taking contact with ARJO Parts sales always have the following information at hand:

- Correct name of the product. See type label. The serial number of the product. See labels.

Manufacturing country
SE=Sweden

Manufacturing works
E=Eslöv

Year

Week

Serial number ←

SEE0102001

Explanation of the serial number:

Product no
AR31211-EU



Serial no
SEE0102001



Made by Arjo in Eslöv, Sweden
010115

Example of identification label

Serial Number From June 2008

The serial number consists of the following parts:

0806000001

Year Month Consecutive Number

Manufacturing country and Manufacturing Works can no longer be readout.

Modifications made to the equipment without the express acknowledgement of ARJO will invalidate supplier's product liability.


Contents Maxi Twin

Overview	4
A Leg and Chassis	6
B Mast, Jib and Lift	8
C1 Dedicated DPS Spreader Manual	10
C2 Dedicated DPS Spreader Powered.....	12
C3 Dedicated DPS Spreader Scale Powered	14
D Dedicated Loops, small, medium, large	16
E Electric Wiring Schematics	18
F Scale.....	20
G Battery and Battery Charger	22
 Preventive Maintenance Parts/Recommended Spares	24
 Service Articles and Special Tools	25

Overview


Pos	Description	See page
A	Leg and Chassis	6
B	Mast, Jib and Lift	8
C1	Dedicated DPS Spreader Manual	10
C2	Dedicated DPS Spreader Powered	12
C3	Dedicated DPS Spreader Scale Powered	14
D	Dedicated Loops, Small, medium, large	16
E	Electric Wiring Schematics	18
F	Scale	20
G	Battery and Battery Charger	22

Overview

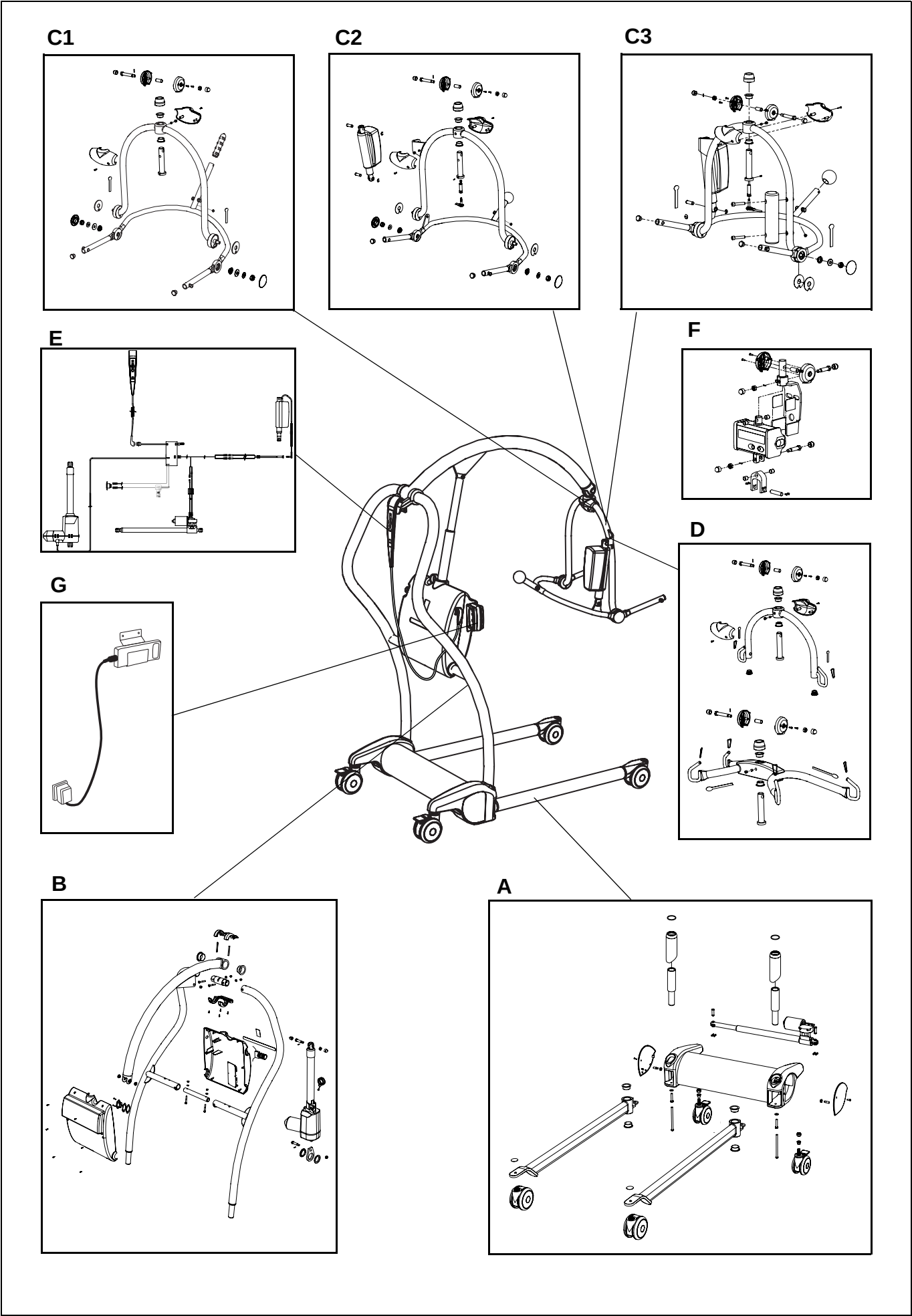


A <i>Leg and Chassis</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
SKTA82645-20		Leg Assembly standard- incl. pos. 1-4:				
1	KTX82587-20	2	Leg standard			
2	KTX00290.0	4	Bush			
3	KMX81702-20	2	Castor standard, 100mm			With thread lock
4	KPX11430.20	2	Cap, front castor			
SKTA82644-20 S8559972-20		Chassis Assembly - which includes pos. 5-18: Chassis Assembly Scale - which includes pos. 5-8, 10-22:				
5	S8259373-031	1	Chassis assy (incl pos 10 and 11)			
6	KTX82505-20	1	End cover, right			
7	KTX82504-20	1	End cover, left			
8	6200005	2	Washer, BRB 8,4x17 A4 DIN 125			
9	6100892	2	Setscrew, M6S M8x40, 10.9			
10	8459278 8460420	2	Setscrew without thread lock 28 mm Setscrew without thread lock 33 mm			For old chassis KTX82832-20 For new chassis 8259373-031
11	6210014	2	Nut M6M M8 A4 DIN 934			
12	KMX81703-20	2	Castor, braked, 100mm			With thread lock
13	KTX82634-00	2	Bush			For old chassis KTX82832-20
14	6210229	2	Locking nut M12 A4, NV 18mm			
15	KMX82333-00	1	Actuator			
16	KPX11400.0	2	Clevis pin, 10 dia			
17	JX68800.0	2	Safety fastner			
18	6190060	2	Screw LKCS 6x12 FN MSG			
19	6100891	2	Screw M6S M8x140, 10.9			Maxi with scale
20	8359731-20	2	Mast extension			Maxi with scale
21	8359735	2	Cover			Maxi with scale
22	6261254	2	O-ring			Maxi with scale

A

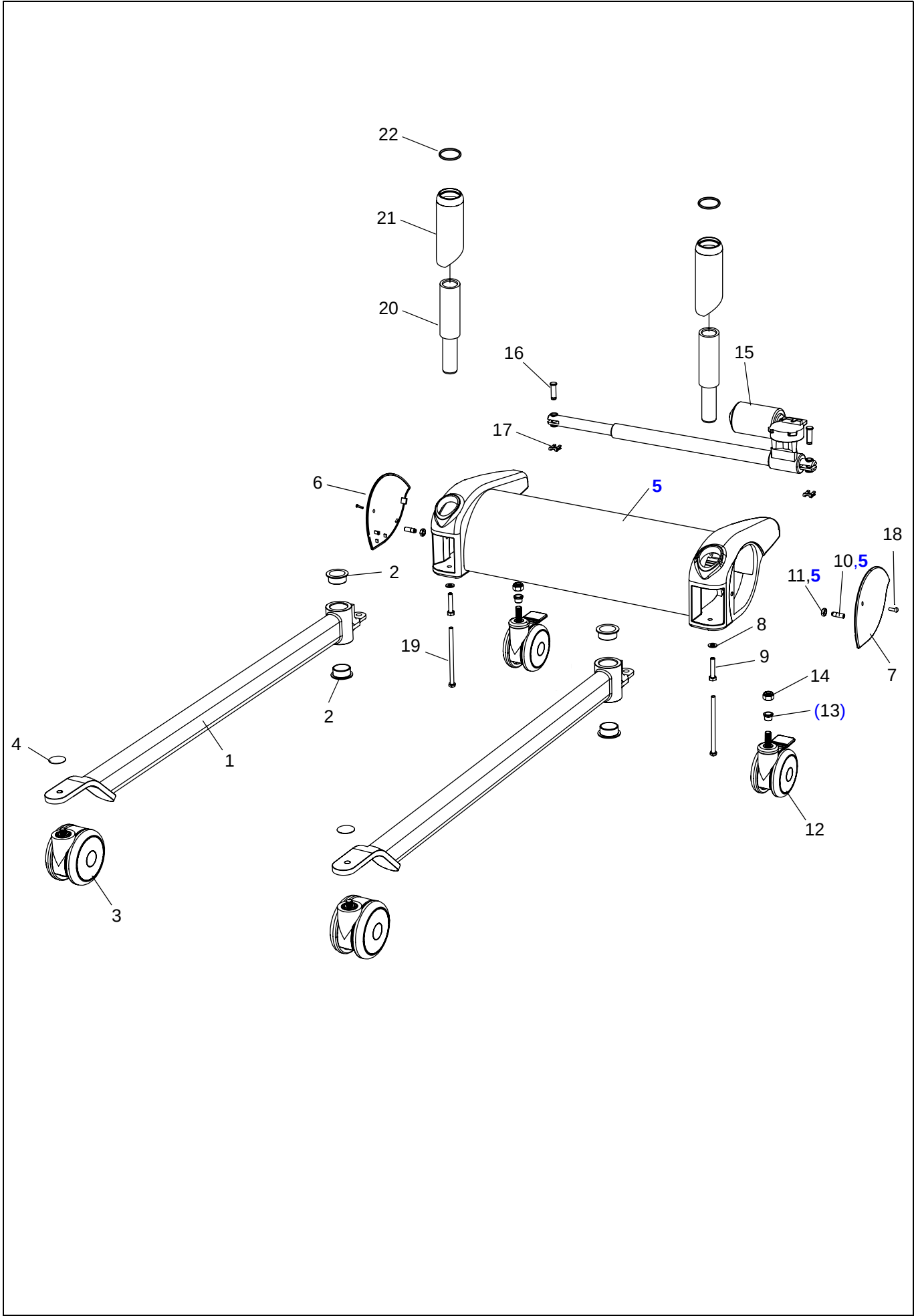

B Mast, Jib and Lift						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
	KTA82649-20	1	Mast, Lift and Jib Assembly - which includes pos. 1-42:			
1	KTX82415-20	1	Jib assembly			
2	KTX82422-00	2	Bearing			
3	KTX82759-00	2	Bearing			
4	KTX82760-00	1	Bearing			
5	KTX82403-20		Mast assembly			
6	6100645	2	Bolt, M6S 6x45 A4 DIN 931			
7	6210043	4	Nut, M6 locking			
8	KTX82585-00	2	Spirol pin, 6 dia x 55 lg			
9	KTX82493-26	1	Handset holder, bottom			
10	KTX82494-26	1	Handset holder, top			
11	6190542	12	Screw RX-PT 4-20x10, A4			
12	6379241	1	Actuator, LA34	SEE0711041		Was KTX82664-00
13	KTX82424-00	1	Actuator extension bracket			
14	KTX82431-00	2	Bearing			
15	KTX82491-26	1	Emergency release			
16	KTX82745-00	2	Pivot bolt			
17	KTX82746-00	2	Pin, 2 dia x 10 lg			
18	6210044	2	Nut locking M8 A4 DIN 985			
19	KTX82756-20	2	Hex cap			
20	KTX82418-24	1	Motor casing, back			
21	8458622	1	Socket			
22	8451301	1	Seal			
23	8451284	1	Clamp plate			
24	6190245	2	Screw, MFX-Z M3x16 A4 ISO 7046			
25	KTX82419-24	1	Motor casing, front			
26	6370605	1	PCB			
27	See E:2	1	Power input loom			New version 0809002109
28	KTX82692-00	1	Label - product name			
29	8418237	1	Label - ARJO			
30	KTX82840-00	2	Screw, M6 x 50 lg, csk			
31	6200003	4	Washer, BRB 6,4x12 A4 DIN 125			
32	6379136	1	Fuse F10AL, 6x32 mm			
33	6379050	1	Fuse T4AL, 5x20 mm			
34	6319516	1	Button			New version 0809002109. On old version also connector must be replaced See E:2.
35	See E:15	1	Handset connector			Not illustrated
36	6370608	4	Attachment short	SEE0644113		
37	6370609	2	Attachment long	SEE0644113		
38	See E:3	1	Battery wire			Not illustrated
39	KRX05070.0	1	Label emergency lowering			Not illustrated
40	See E:5	1	Powered DPS loom 1*			Not illustrated
41	See E:6	1	Powered DPS loom 2*			Not illustrated
42	8418626	1	Data label			Not illustrated
43	8547035	1	Kit Emergency stop			Including pos 27 and pos 34


B

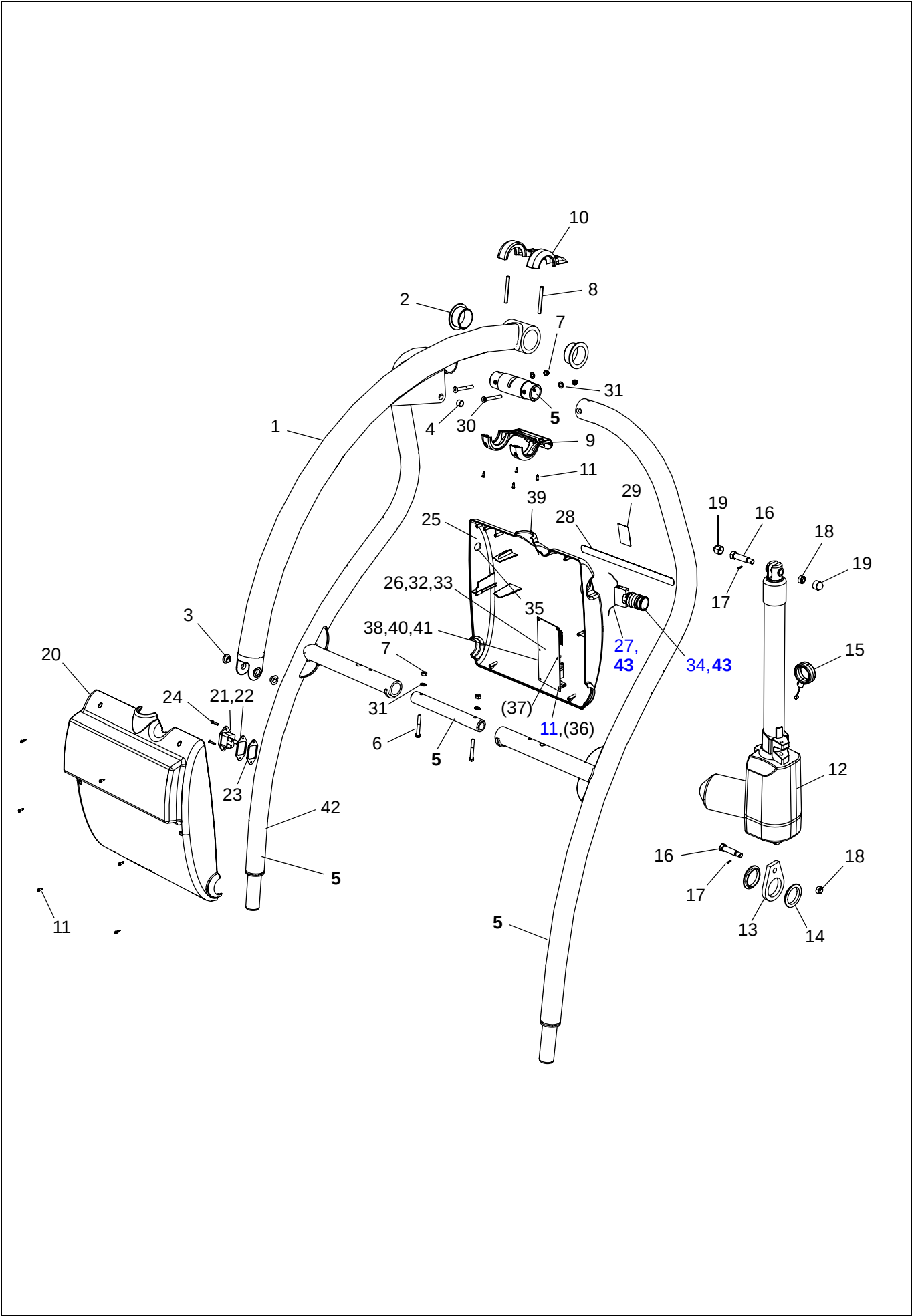

<i>C1</i>		<i>Dedicated DPS Spreader Manual</i>				
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
	SKTA82657-24		DPS Assembly - which includes pos. 1-27:			
1	KTX82438-24	1	Upper lift frame			
2	KTX82421-00	2	Bearing			
3	KTX82477-00	1	Vertical pin			
4	KTX82701-20	1	Pin cover			
5	6306148	1	Bearing			
6	8359446-20	1	Cover, jib left			
7	8359447-20	1	Cover, jib right			With screw holes
8	6190542	2	Screw RX-PT 4-20x10, A4			
9	8459511	1	Slug			
10	KTX82703-00	1	Screw, M8 x 6, skt set			
11	KMX82671-00	2	Friction washer			
12	KTX82481-24	1	Lift frame			
13	KPX67540.0	2	Glacier bush			
14	GX20460.0	2	Washer spring disc			
15	MX01450.0	2	Washer			
16	6210045	2	Nut locking, M10 A4 8.8 DIN 985			
17	MX01460.20	2	Cap			
18	KPX20100.20	1	Hand grip			
19	KPX20310.0	2	End plug			
20	KTX82474-20	2	Cover, spreader bar			
21	KTX82688-00	2	Label hanger, SWL 170kg			
22	KTX82744-00	1	Pivot bolt			
23	6210044	1	Nut locking M8 A4 DIN 985			Locking
24	KTX82746-00	1	Pin, 2 x 10 LG			
25	KTX82756-20	2	Hex cap			
26	6190544	2	Screw RX-TP 4-20x16, fzb			
27	KMX01080.0	1	Plug			
28	6319525	1	Label slings			5 labels in one


C2		Dedicated DPS Spreader Powered				
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
	SKTA82659-24		Powered DPS Assembly - which includes pos.1-34:			
1	KTX82480-24	1	Upper lift frame			
2	KTX82421-00	2	Bearing			
3	KTX82477-00	1	Vertical pin			
4	KTX82701-20	1	Pin cover			
5	6306148	1	Bearing			
6	8359446-20	1	Cover, jib, left			
7	8359447-20	1	Cover, jib right			
8	6190542	2	Screw RX-PT 4-20x10, A4			
9	8459511	1	Slug			
10	KTX82703-00	1	Screw, M8 x 6, skt set			
11	KPX67640.0	4	Friction washer			
12	KTX82483-24	1	Lift frame			
13	KPX67540.0	2	Glacier bush			
14	6190538	1	Set screw, Triloc 211			
15	AX05700.0	2	Washer			
16	6210045	2	Nut locking, M10 A4 8.8 DIN 985			
17	MX01460.20	2	Cap			
18	KPX67120.20	1	Handgrip knob			
19	KPX20310.0	2	End plug			
20	KTX82474-20	2	Cover, spreader bar			
21	KTX82688-00	2	Label hanger, SWL 170kg			
22	KTX82744-00	1	Pivot bolt			
23	6210044	1	Nut locking M8 A4 DIN 985			
24	KTX82746-00	1	Pin, 2 x 10 LG			
25	KTX82756-20	2	Hex cap			
26	See E:8	1	Socket and wire assy			
27	See E:7	1	Powered DPS loom 3			
28	SKTX82665-00	1	Actuator, LA28			Incl. pos. 29-30
29	KPX67200.0	2	Pin			
30	KPX67210.0	2	'E' clip			
31	LBX05550.0	1	Label water level			
32	KMX01080.0	1	Plug			
33	6190544	2	Screw RX-TP 4-20x16, fzb			
34	KMX82687-00	1	Heatshrink			
35	8547002	1	Jack plug/socket kit			Incl pos. 3,5,9,14,23,26 and 27
36	6319525	1	Label slings			5 labels in one

C2

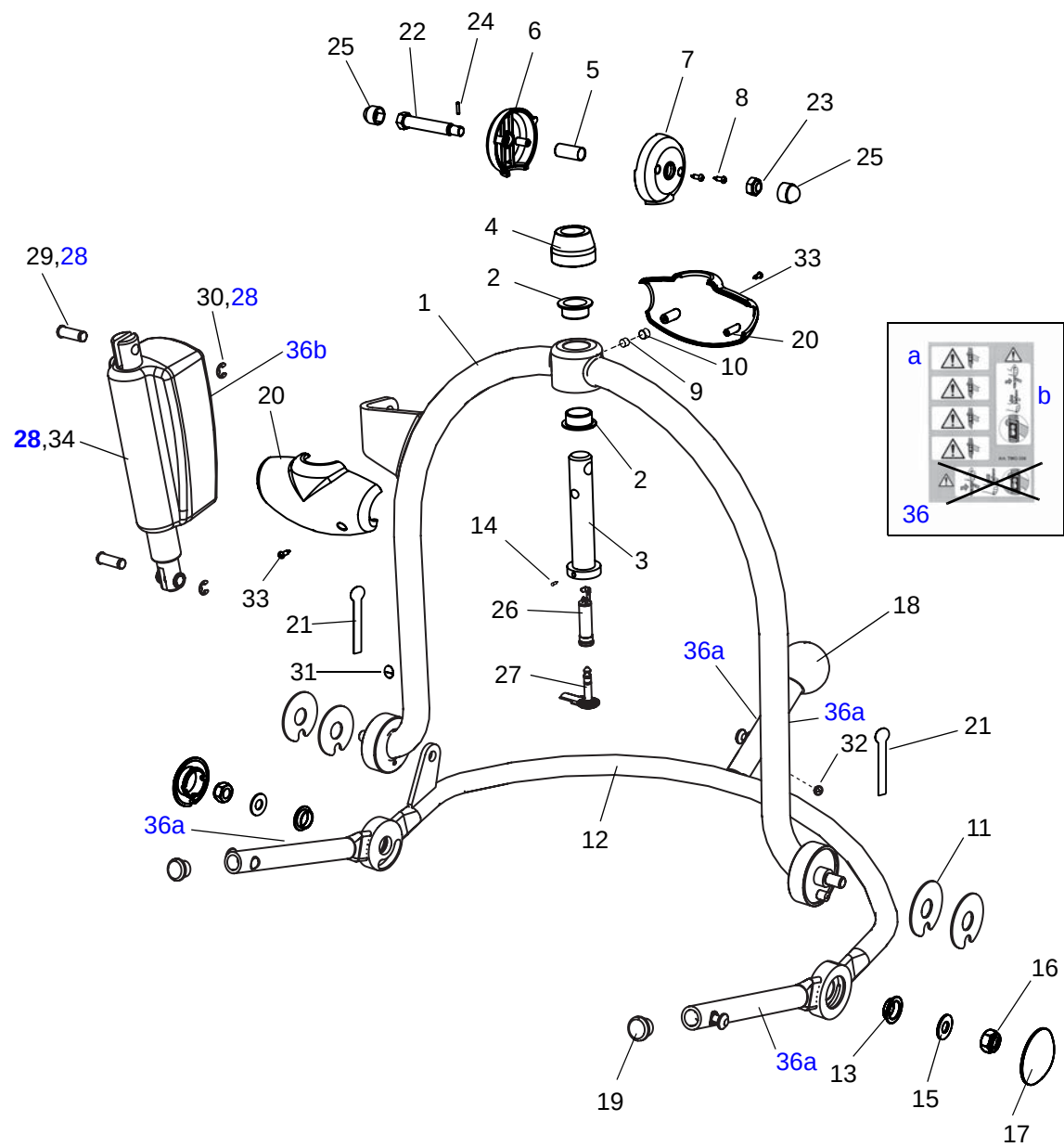



C3		Dedicated DPS Spreader Scale Powered				
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
S8559973-24		Powered DPS Assembly - which includes pos.1-36:				
1	8159975	1	Upper lift frame			
2	KTX82421-00	2	Bearing			
3	KTX82477-00	1	Vertical pin			
4	KTX82701-20	1	Pin cover			
5	6306148	1	Bearing			
6	8359930-20	1	Cover, jib,left			
7	8359931-20	1	Cover, jib right			
8	6190542	2	Screw RX-PT 4-20x10, A4			
9	8459511	1	Slug			
10	KTX82703-00	1	Screw, M8 x 6, skt set			
11	KPX67640.0	4	Friction washer			
12	KTX82483-24	1	Lift frame			
13	KPX67540.0	2	Glacier bush			
14	6190538	1	Set screw, Triloc 211			
15	AX05700.0	2	Washer			
16	6210045	2	Nut locking, M10 A4 8.8 DIN 985			
17	MX01460.20	2	Cap			
18	KPX67120.20	1	Handgrip knob			
19	KPX20310.0	2	End plug			
20	KTX82474-20	2	Cover, spreader bar			
21	KTX82688-00	2	Label hanger, SWL 170kg			
22	KTX82744-00	1	Pivot bolt			
23	6210044	1	Nut locking M8 A4 DIN 985			
24	KTX82746-00	1	Pin, 2 x 10 LG			
25	KTX82756-20	2	Hex cap			
26	See E:8	1	Socket and wire assy			
27	See E:7	1	Powered DPS loom 3			
28	SKTX82665-00	1	Actuator, LA28			Incl. pos. 29-30
29	KPX67200.0	2	Pin			
30	KPX67210.0	2	'E' clip			
31	LBX05550.0	1	Label water level			
32	KMX01080.0	1	Plug			
33	6190544	2	Screw RX-TP 4-20x16, fzb			
34	KMX82687-00	1	Heatshrink			
35	8359732-24	1	Counter weight			
36	6130840	2	Screw MC6S 8x40 A4 DIN 912			
37	8547002	1	Jack plug/socket kit			Incl pos. 3,5,9,14,23,26 and 27
38	6319525	1	Label slings			5 labels in one

C3

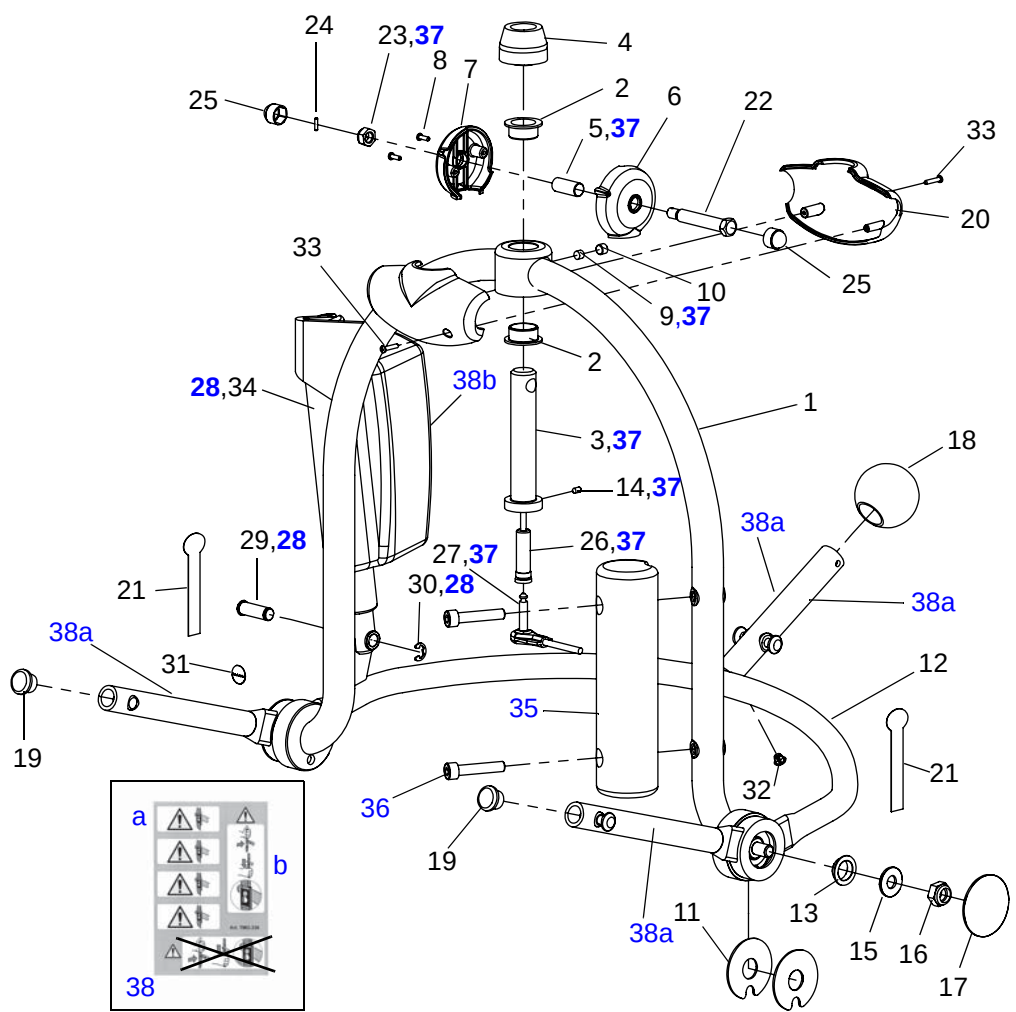


D <i>Dedicated Loops (small, medium and large)</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
	SKTA82846-24		Loop, Small, Dedicated, assembly (not illustrated) pos. 1-19			
	SKTA82845-24		Loop, Medium, Dedicated, assembly - which includes: pos. 1-19			
1	KTX82264-24 KTX82265-24	1	Loop, small, welded assy Loop, medium, welded assy			not illustrated illustrated
2	KTX82421-00	2	Bearing			
3	KTX82477-00	1	Vertical pin			
4	KTX82701-20	1	Pin cover			
5	6306148	1	Bearing			
6	8359446-20	1	Cover, jib left			Without screw holes
7	8359447-20	1	Cover, jib right			With screw holes
8	KTX82744-00	1	Pivot bolt			
9	6210044	1	Nut locking M8 A4 DIN 985			
10	6190542	2	Screw RX-PT 4-20x10, A4			
11	KTX82756-20	2	Hex cap, M8			
12	8459263	2/4	Retaining clip			Loop (S,M) 2, Loop (L) 4
13	KTX82746-00	1	Pin, 2 x 10 LG			
14	6190544	2	Screw RX-TP 4-20x16, fzb			
15	KTX82703-00	1	Screw, M8 x 6, skt set			
16	8459511	1	Slug			
17	KTX05210.0	2	End plug			Loop (S,M)
18	KTX82474-20	2	Cover, spreader bar			Loop (S,M)
19	KTX82688-00	2	Label hanger, SWL 170kg			Loop (S,M)
	SKTA82847-24		Loop, Large, Dedicated, assembly - which includes pos 2-16, 20-21:			
20	KTX82554-24	1	Loop, large, welded assy			Loop (L)
21	KTX82689-00	2	Label loop large, SWL 170kg			Loop (L)

D

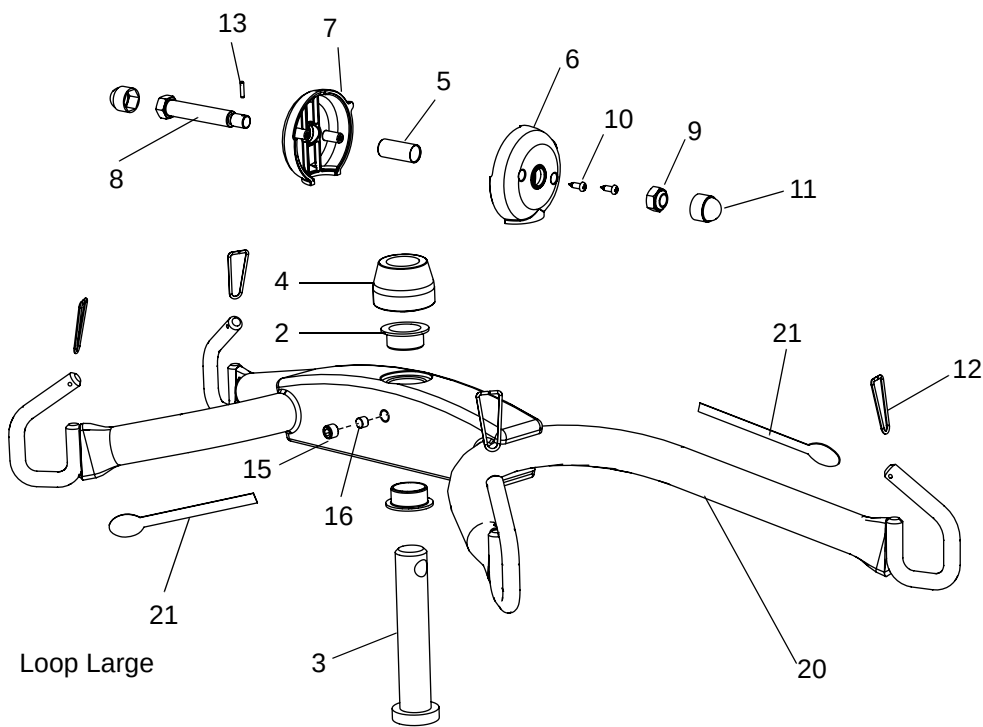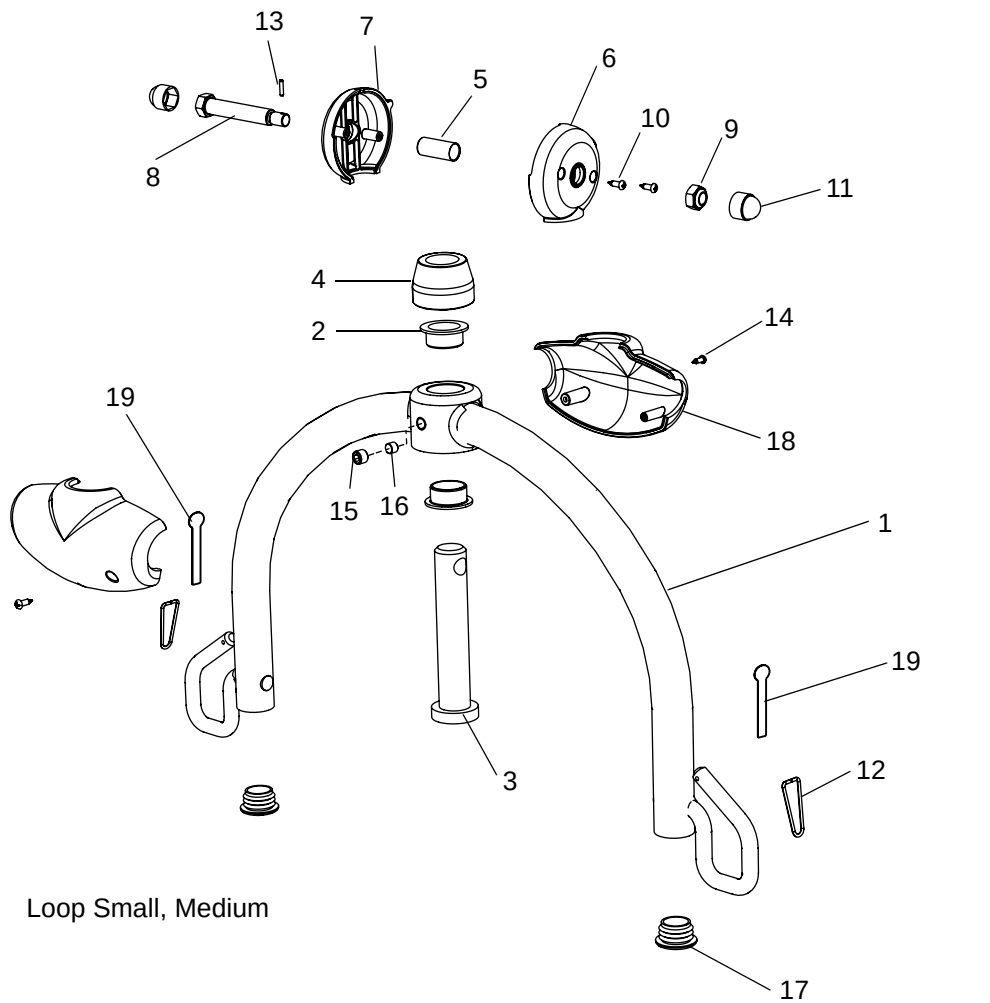


<i>E Electric Wiring Schematics</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	See B:26	1	PCB			
2	8360696	1	Power input loom			New version 0809002109. On old version also button must be replaced. See B:34
3	8358604	1	Battery wire			
4	8358600	1	Leg loom 2			
5	8358599	1	Powered DPS and leg loom 1			
6	8358598	1	Powered DPS loom 2			
7	8358597	1	Powered DPS loom 3			
8	8359836	1	Socket & wire assy			Extended version
9	See B:21	1	Socket			
10	See B:12	1	Lift actuator, LINAK LA34			
11	See C2:28	1	Powered DPS actuator,			
12	See A:17	1	Leg actuator, single ended			
13	8560020	1	6 function handset			
14	8560016	1	4 function handset			
15	8358605	1	Handset connector			
16	8358603	1	Bridge connector			

E

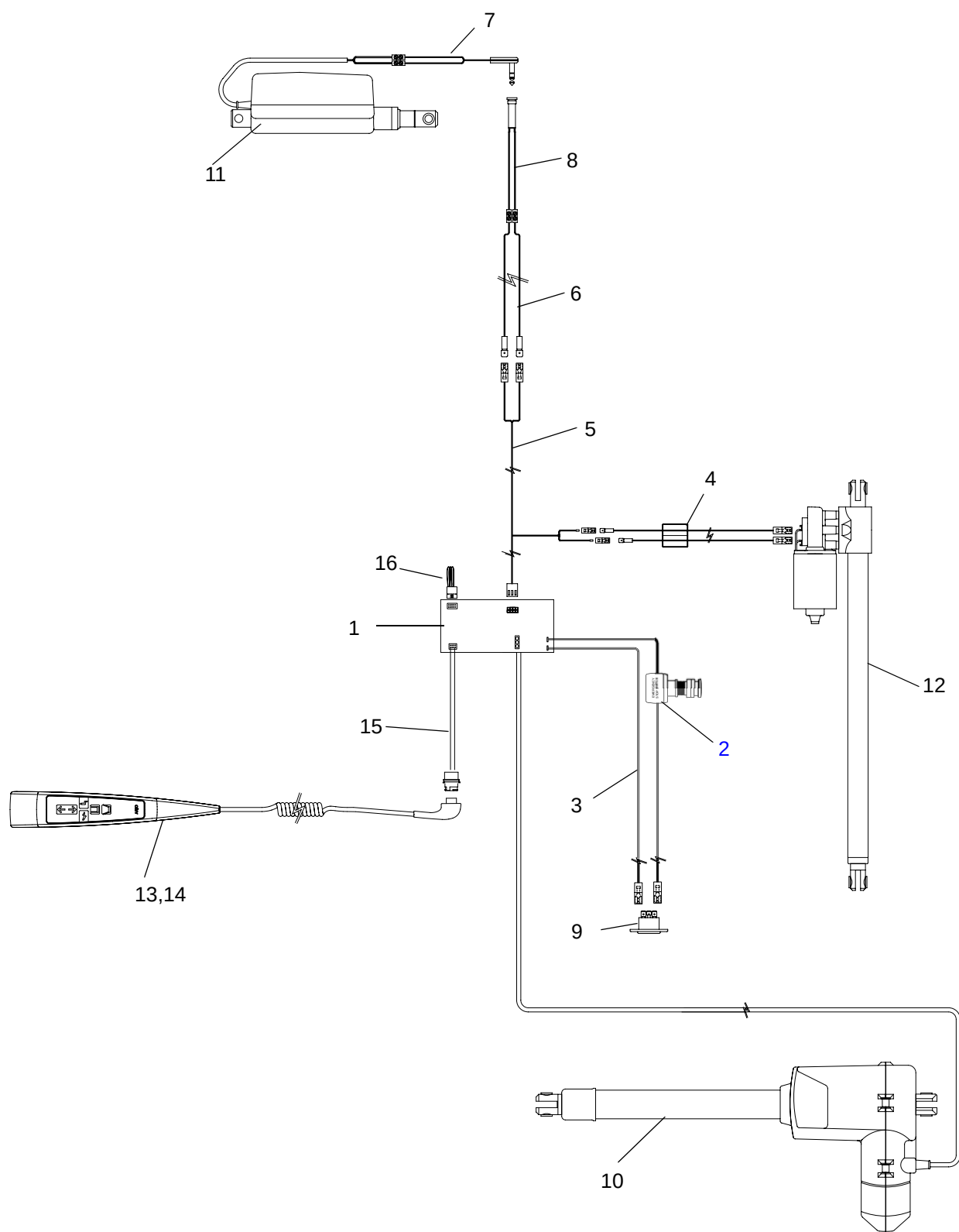


<i>F</i>		<i>Scale</i>				
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	CFA3000-03	1	Maxi Twin Scale			
2	8259729-20	1	Scale bracket			
3	8459730	2	Friction plate			
4	8259726	1	Fork			
5	8459727	1	Fork axle			
6	6190562	4	Screw MFX 3x6 A4			
7	6306149	4	Bearing			
8	6306148	1	Bearing			
9	KTX82745-00	2	Pivot bolt			
10	KTX82746-00	2	Pin, 2 x 10 LG			
11	6210044	2	Nut locking M8 A4 DIN 985			
12	KTX82756-20	4	Hex cap			
13	8359737-20 8359446-20	1	Cover, jib,left with Power DPS Cover, jib,left			
14	8359447-20	1	Cover, jib right			With screw holes
15	6190542	2	Screw RX-PT 4-20x10, A4			
16	6320233	1m	Adhesive tape 3M VHB 4905F			
17	6907940	1	Battery holder			

F

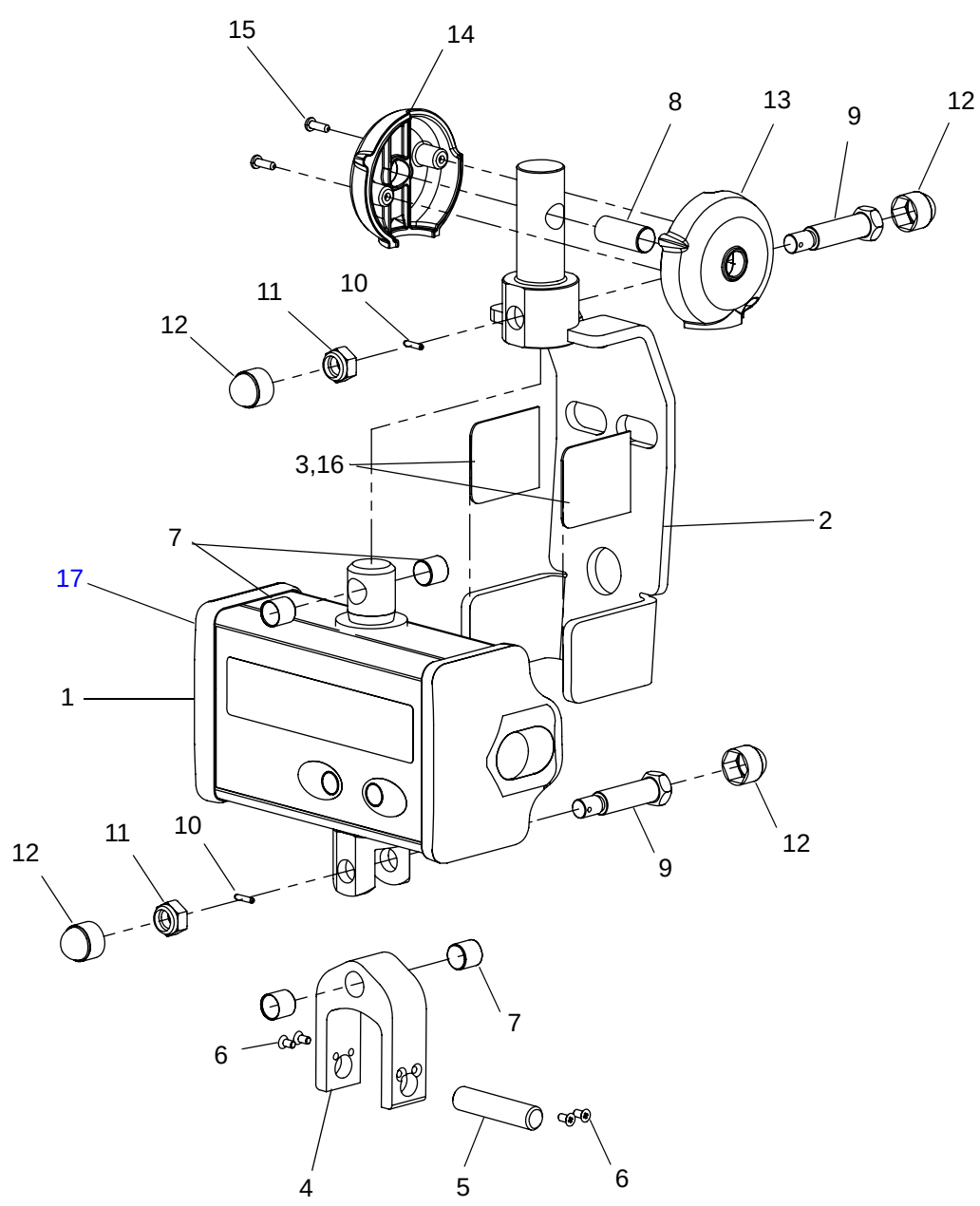

G Battery and Battery Charger						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	NEA 0100-083	1	Battery			
2	8350891-031	1	Battery Holder			
3	NEA 1000-EU NEA 2000-GB NEA 4000-US NEA 6000-AU NEA 7000-JP	1	Battery Charger			

G

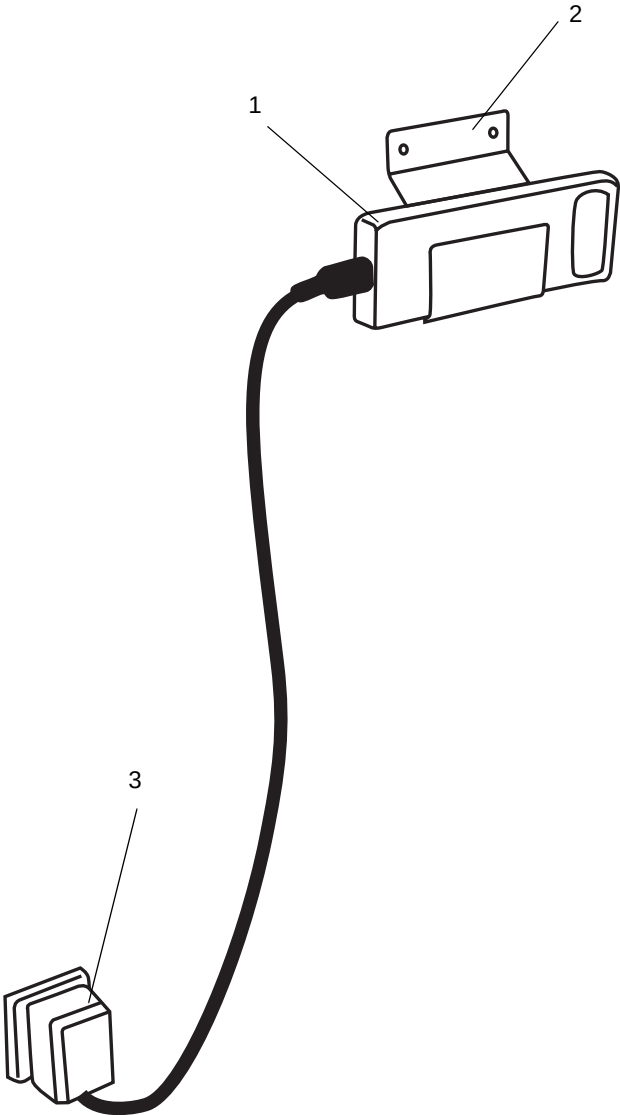

Maxi Twin

Preventive Maintenance Parts/Recommended Spares

Preventive Maintenance Parts	Part number	Number of Maxi Twin*		Page
		1	5	
Castor standard, 100mm	KMX81702-20	2	10	A:3
Castor, braked, 100mm	KMX81703-20	2	10	A:12

Recommended Spares	Part number	Number of Maxi Twin*		Page
		1	5	
4 function handset	8560016	1	2	E:14
6 function handset	8560020	1	2	E:13
Actuator, LA34	KTX82664-00	1	1	B:12
Battery	NEA 0100-083	1	2	G:1
Bearing	KTX82431-00	2	4	B:14
Bearing	KTX82421-00	2	4	C:2,D:2
Bearing	6306148	1	2	C:5,D:5,F:8
Cap	KTX82756-20	2	4	B:19,C:25, D:11
Clevis pin, 10 dia	KPX11400.0	1	1	A:16
Cover, Jib right (With screw holes)	8359447-20	1	2	C1:7,C2:7, D:7,F:14
Cover, Jib left	8359446-20	1	2	C1:6,C2:6, D:6
Cover, Jib left with Power DPS	8359737-20	1	2	F:13
Cover, Jib left with Power DPS and scale	8359930-20	1	2	C3:6
Cover, Jib right with Power DPS and scale	8359931-20	1	2	C3:7
Cover, spreader bar	KTX82474-20	2	4	C1:20,C2:20 D:18
Fuse	6379136	1	2	B:32
Fuse	6379050	1	2	B:33
Handset holder, bottom	KTX82493-26	1	2	B:9
Handset holder, top	KTX82494-26	1	2	B:10
Nut locking M8 A4 DIN 985	6210044	3	6	B:18 C:23 D:9 F:11
PCB	6370605	1	1	B:26
Pin, 2 x 10 LG	KTX82746-00	2	4	B:17,C:24 D:13
Pivot bolt, actuator	KTX82745-00	2	4	B:16,F:9
Pivot bolt, jib	KTX82744-00	1	2	C:22,D:8
Powered DPS loom 3	8358597	1	2	E:7
Retaining clip (Not for DPS)	8459263	4	20	D:12
Safety fastner	JX68800.0	2	4	A:17
Vertical pin	KTX82477-00	1	1	C:3,D:3

*For larger numbers of Maxi Twin than 5 add the same number of parts as between 1 and 5 for every increase of 5 Maxi Twin.
Note. If the same quantity is recommended for 1 and 5 Maxi Twin's no addition should be done before an increase of 15.

Maxi Twin

Service Articles and Special Tools

Service Articles	Type	Quantity	Part number
Shell Cassida	Lubrication	400ml spray	6907754
Loctite 243	Locking	10ml	6950607
Loctite 495	Glue	20g	6311090
Essve 540	Glue		6907941
Touch-up paint	Grey	20ml	6951007-031
Touch-up paint	Light blue	20ml	6951007-024
Touch-up paint	Dark blue	20ml	6951007-31

Special Tools	Part number
Angle Plate	04063
Leg Measurement Tool	04064
Logger	8559816
Sling Clip Gauge	ST331
Special Tool	ST252
Load Test Equipment: Eye Hook	ST240
Load Test Equipment: Handset and Loadcell	ST164
Load Test Equipment: Hinge Beam	ST239
Load Test Equipment: Load Cell Housing	ST241
Load Test Equipment: Strap	2xST171
Load Test Equipment: Strap (long)	2xST166
Load Test Equipment: Strap (short)	2xST165

AUSTRALIA

ARJO Hospital Pty Ltd
Unit 1, 205 Queensport Road
Murrarie
QLD 4172
E-mail: info@arjo.com.au
Tel: 07-3395 6311
Faxnr. 07-3395 6712

AUSTRIA

ARJOHuntleigh GmbH
Dörrstrasse 85
A-6020 Innsbruck
E-mail: office@arjo.de
Tel: 0512 204 160-0
Faxnr. 0512 204 160-75

BELGIUM

ARJOHuntleigh NV/SA
Evenbroekveld 16
BE-9420 Erpe-Mere
E-mail: info@arjohuntleigh.be
Tel: 053 60 73 80
Fax: 053 60 73 81

CANADA

ARJO Canada Inc.
1575 South Gateway Road
Unit C
Mississauga, Ontario, L4W 5J1
E-mail: info@arjo.ca
Tel: 1-800-665-4831
Fax: 1-800-309-1116

CZECH REPUBLIC

ARJO Hospital Equipment S.R.O.
Hlinky 118
CZ- 603 00 Brno
E-mail: arjo@arjo.cz
Tel.: 549 254 252
Fax: 541 213 550

DENMARK

Huntleigh Healthcare A/S
Vassingerødvej 52
DK-3540 Lynge
E-mail: info@huntleigh-healthcare.dk
Tel: 49 13 8486
Fax: 49 13 8487

FAR EAST

ARJO Far East Ltd
Unit 3A, 4/F., Block B Hoi Luen Industrial Centre
55 Hoi Yuen Road, Kwun Tong,
Kowloon, Hong Kong
E-mail: samuelw@arjo.com.hk
Tel: 2508 9553
Fax: 2389 5797

FINLAND

OY Vestek AB
Martinkuja 4
02270 Espoo
E-mail: vestek@vestek.fi
Tel: 9 8870120
Fax: 9 88701291

FRANCE

ARJO Equipements Hospitaliers S.A.
45, Avenue de l'Europe
Eurocit BP 133
Ronsq Cedex, FR-59436
E-mail: info@arjo.fr
Tel: 03 20 28 13 13
Fax: 0 3 20 28 13 14

GERMANY

ARJOHuntleigh GmbH
Peter-Sander-Strasse 10
DE-55252 Mainz-Kastel
E-mail: info@arjo.de
Tel: 06134-186-0
Fax: 06134-186-160

GREECE

C. Psimitis Co Ltd
59, Dimitrion Str.
16121 Kaisariani, Attiki
Tel: 210 724 36 68
Fax: 210 721 55 53

ITALY

ARJOHuntleigh S.p.A.
Via Tor Vergata 432
IT-00133 Roma
E-mail: promo@arjohuntleigh.it
Tel: 06-87426211
Fax: 06-87426222

THE NETHERLANDS

ARJOHuntleigh Nederland B.V.
De Blomboogerd 8
4003 BX TIEL
Postbus 6116
4000 HC Tiel
E-mail: info@arjohuntleigh.nl
Tel: 0344-64 08 00
Fax: 0344-64 08 85

NORWAY

ARJO Scandinavia
Ryenstubben 2
NO-0679 Oslo
E-mail: kundservice@arjo.se
Tel: 98 28 11 70
Fax: 22 57 06 52

POLAND

ArjoHuntleigh Polska Sp. z o.o.
ul. Ks Wawrzyniaka 2 w
PL 62-052 KOMORNIKI
Tel. +48 61 662 15 50
Fax +48 61 662 15 90
info@arjohuntleigh-polska.com

PORTUGAL

ARJO International AB

PT 4510-154 JOVIM GONDOMAR
Tel: 22 450 32 48
Fax: 22 450 03 27

SPAIN

ArjoHuntleigh Ibérica S.A
Ctra. de Rubí, 88 1ª planta - A1
08173 Sant Cugat del Vallés, BARCELONA
E-mail: info@arjohuntleigh.es
Tel: 93 583 1120
Fax: 93 583 1122

SWEDEN

ARJO Scandinavia AB
Verkstadsvägen 5
Box 61
SE-241 21 Eslöv
E-mail: kundservice@arjo.se
Tel: 0413-645 00
Fax: 0413-645 83

SWITZERLAND

ArjoHuntleigh AG
Florenzstrasse. 1d
Postfach
CH-4023 Basel
E-mail: arjo-sic@arjo.ch
Tel: 061-337 97 77
Fax: 061-311 97 42

UK

ARJO Med AB Limited
St Catherine Street
Gloucester GL1 2SL
E-mail: uksales@arjo.co.uk
Tel: 08702 430 430
Fax: 01452-428 344

USA

ARJO Inc.
50 North Gary Avenue
Roselle, IL 60172
E-mail: info@arjousa.com
Tel: 1-800-323-1245
Fax: 1-888-594-2756



FM 24321 ISO 9001
MD 87841 ISO 13485



7000202

ARJO

...with people in mind.

www.arjo.com

info@arjo.com

MEMBER OF THE GETINGE GROUP

If your country is not listed here, please contact your local distributor or:
ARJO International AB, Box 61
SE-241 21 Eslöv, SWEDEN, E-mail: arjo.international@arjo.com
Tel: +46 413 645 00; Fax: +46 413 138 76