


Sara 3000TM

ARJO



Parts list

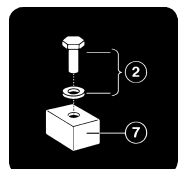

KKX81208-00-M
issue 2
May 2007

Introduction

© ARJO

ARJO products are patented or patent pending.

Patent information is available by contacting ARJO.

The policy of Arjo is one of continuous development and we therefore reserve the right to make technical alterations without notice.

The content of this publication may not be copied either whole or in part without the consent of ARJO.

This product has been manufactured for ARJO by :
Medibo Medical Products N.V.
Heikant 5
B-3930 Hamont-Achel
Belgium

'SARA 3000' and 'ARJO' are registered trademarks of GETINGE AB.

The SARA 3000 is produced in Belgium by
MEDIBO MEDICAL PRODUCTS for GETINGE
AB and sold under the ARJO brand.


Contents

	SECTION	Page No.
A	Chassis assembly.....	6-11
B	Mast assembly	12-19

Final assembly Sara 3000

Item No	Part Number	Description	Qty.
Items on page 5		Assembly Sara 3000	
1	X	See Chassis assembly Sara 3000	X
2	X	See Mast assembly Sara 3000	X
3	HMX94	Plastic foot support	1
4	HMX185	Cap D22 flat	2
5	X	See knee support assembly	X
6	HMX347	M8 x 12 allen bolt	2
7	HMX127	M10 x 20 allen bolt with flange	3

Final assembly Sara 3000

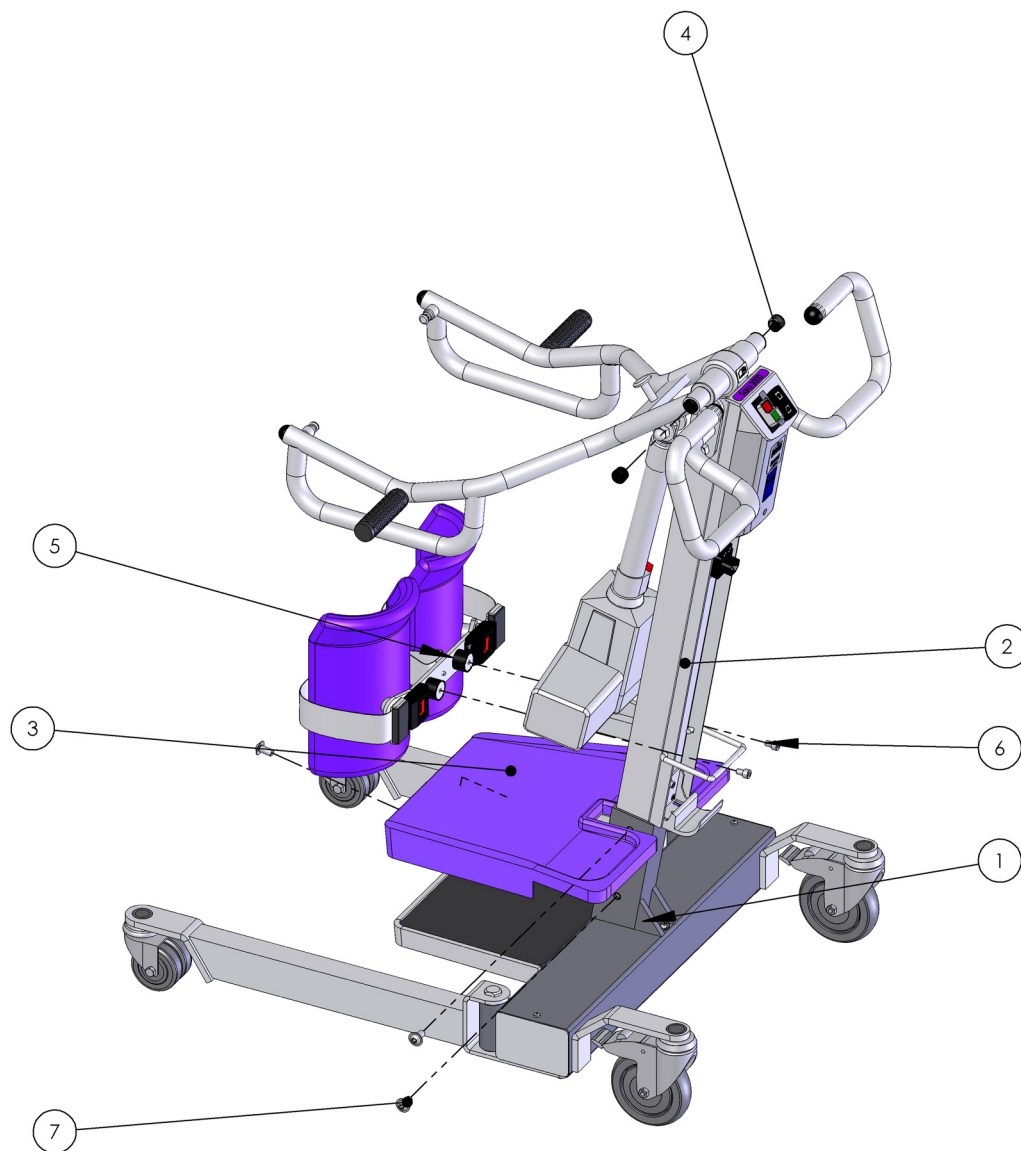


A - Chassis

Item No	Part Number	Description	Qty.
Items on page 8		Chassis	
1	HMX334	Chassis active lift	1
2	HMX699	Distance bushing bearing	2
3	HMX131	Chassis leg bearing	4
4	HMX484	Cover plate chassis	2
5	TGGE.311	Tente wheel 125 braked	2
6	HMX140	M12 x 35 allen bolt	2
7	HMX550	M12 locknut	2
8	HMX345	4.8 mm speednut	2
9	HMX569	Trespa footplate	1
10	HMX44	Linak actuator (part not illustrated)	1
11	HMX291	Bush chassis opening mech.	2
12	HMX128	M8 x 40 allen bolt	2
13	HMX216	M8 locknut	2
14	HMX349	M4.8 x 19 screw	2
15	HMX275	Anti skid cover	2
16	HMX565	M8 x 16 allen bolt	2
17	HMX389	M8 x 20 allen bolt	1
18	HMX1098	Cable assembly chassis	1
19	HMX272	M3 x 6 cylindrical screw	2
20	HMX141	AMP 2p plug female (part not illustrated)	1
Items on page 9		Leg	
1	HMX333	Sara chassis leg	2
2	HMX336-1	Bearing pin	2
3	TGE.310	Tente castor D75 unbraked	2
4	HMX275	Anti skid cover	2
5	HMX127	M10 x 20 allen bolt with flange	4
6	HMX69	M10 x 35 allen bolt	2
7	HMX200	Locknut M10	2
Items on page 10		Knee support	
1	HMX109	Support plate	1
2	HMX124	Hex. bolt M16 x 20	2
3	HMX347	M8 x 12 allen bolt	2
4	HMX218	M4 x 8 cylindrical screw	4
5	see item 8	Belt + 2 metal locks	0
6	HMX931	Knee support	2
7	HMX107	Silence blocks	2
8	HMX522	Belt + 2 metal locks	1
9	see item 8	Belt + 2 metal locks	0
10	HMX433	M6 x 25 washer	1
11	HMX556	M6 x 12 cylindrical screw	1

A - Chassis


Item No	Part Number	Description	Qty.
	Items on page 11	Electronically powered chassis	
1	HMX872	Base plate	1
2	HMX412	Nylon ring M8 thin	10
3	HMX292	Turning disc (connecting part)	1
4	HMX290	Strip (connecting part)	1
5	HMX266	Pushbar el. pow. chassis - straight	1
6	HMX288	Pushbar el. pow. chassis - bent	1
7	HMX287	Movement strip (connecting part)	1
8	HMX119	Ball joint M8 x 30 cw thread	2
9	HMX216	Locknut M8	6
10	HMX152	Hex. bolt M8 x 35	2
11	HMX193	Nylon ring M10	3
12	HMX44	Linak actuator 75 mm	1
13	HMX200	Locknut M10	2
14	HMX214	Nut M8	4
15	HMX439	Spherical rod end	2
16	HMX355	Hex. bolt M10 x 45	1
17	HMX596	AMP plug female (pk25) (part not illustrated)	2
18	HMX144	AMP 2p plug male (part not illustrated)	1

A - Chassis

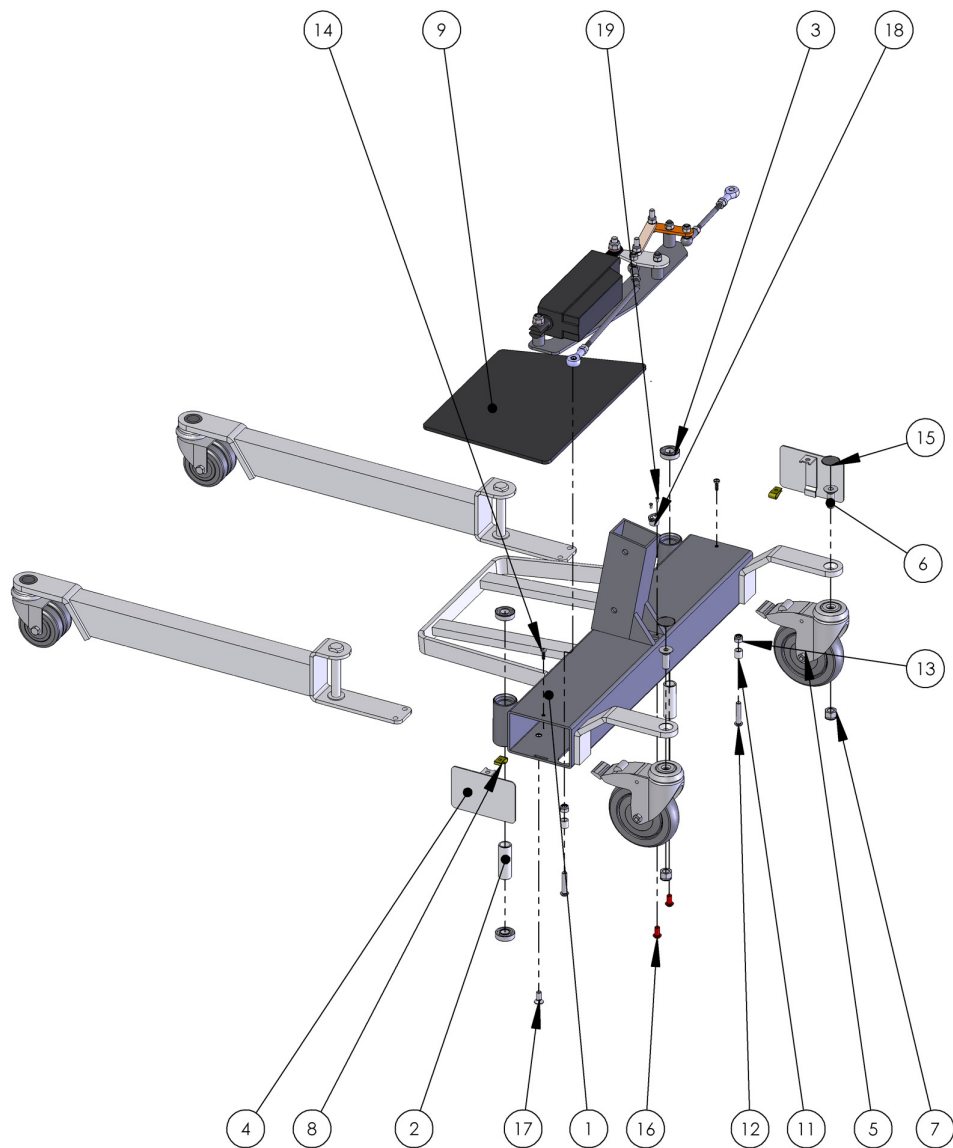


A - Chassis

Leg

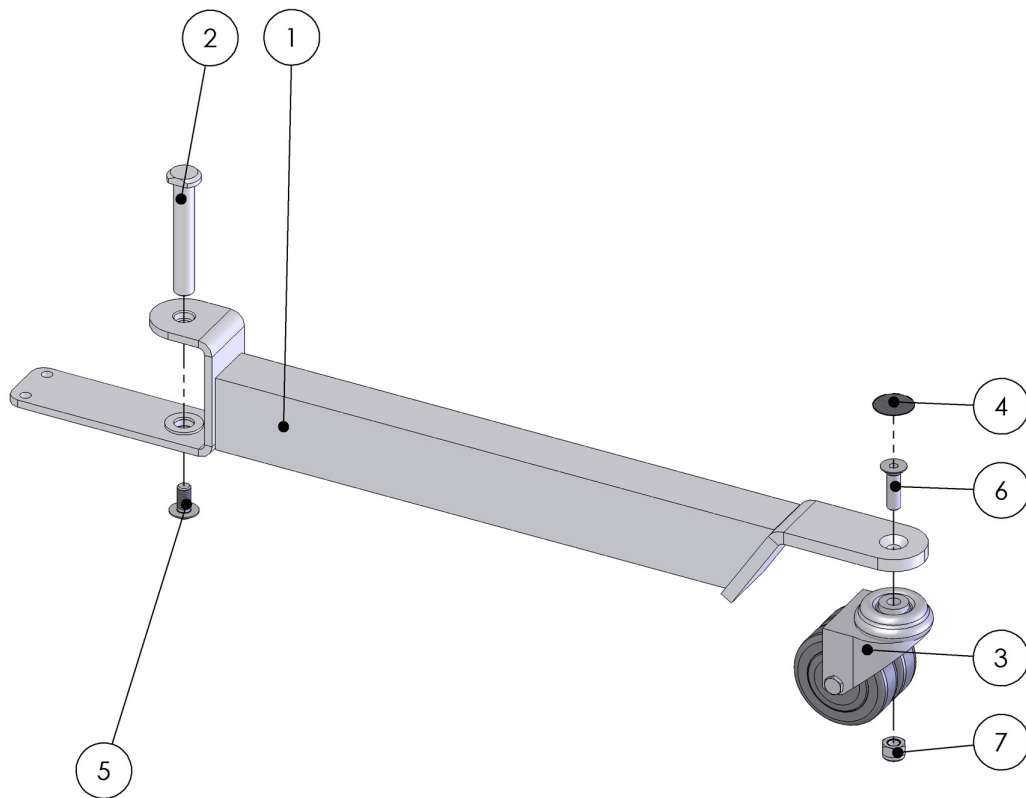


A - Chassis

Knee Support

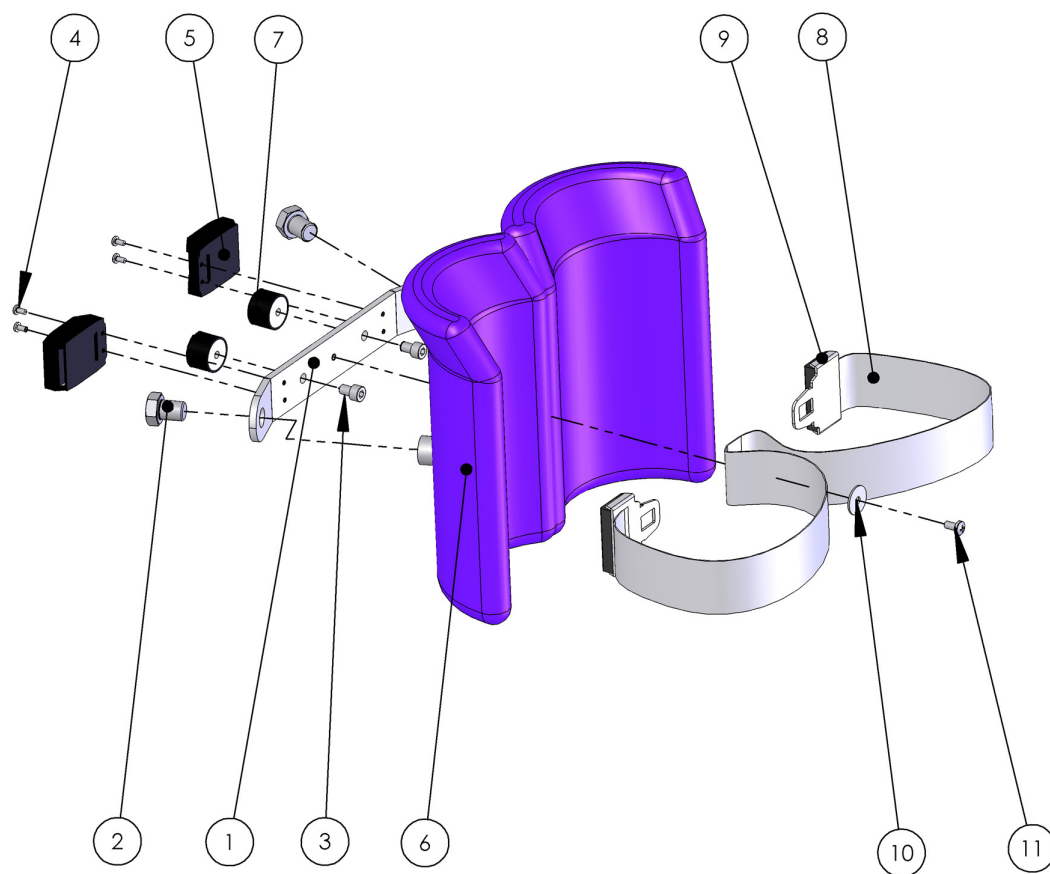

A - Chassis

Electronically powered chassis

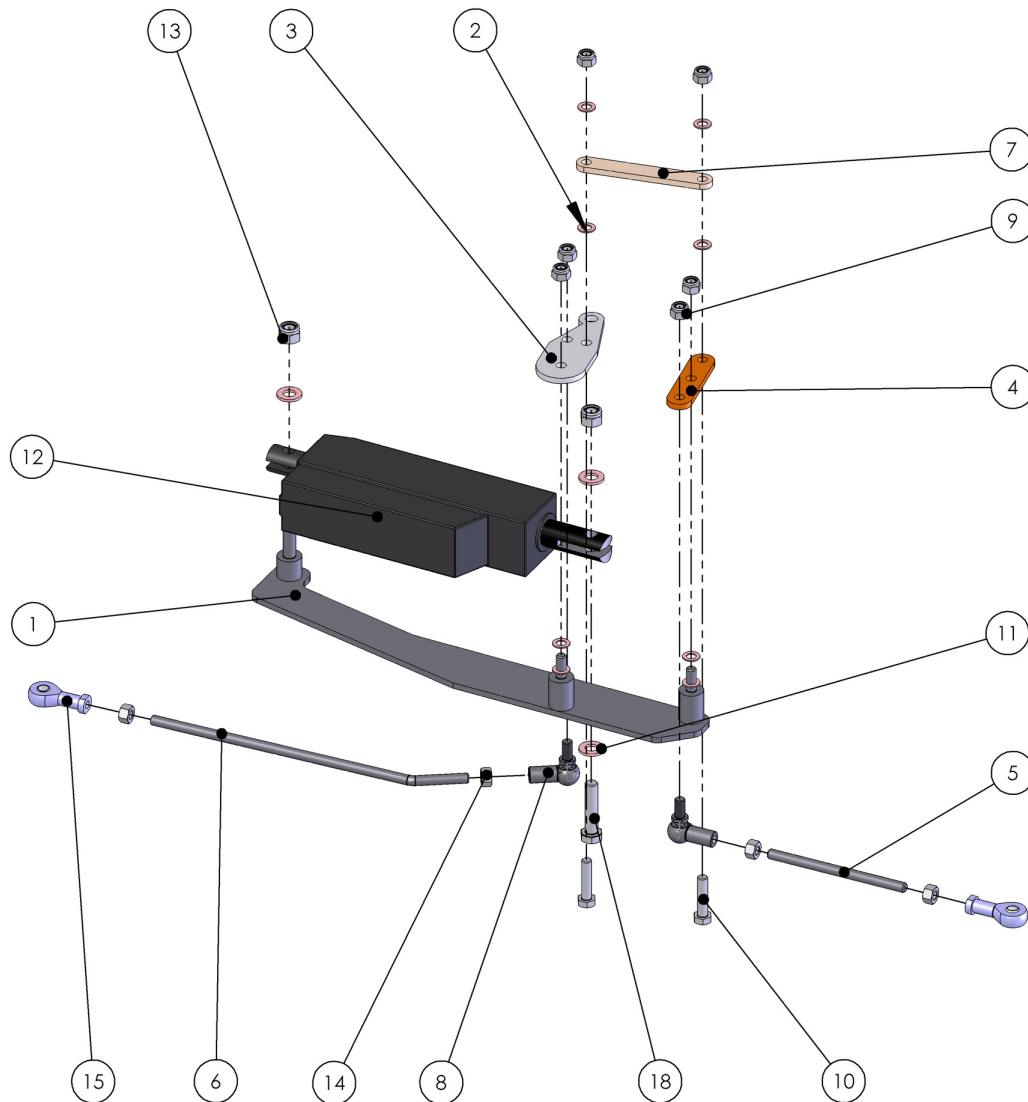


B - Mast

Item No	Part Number	Description	Qty.
Items on page 14		Mast	
1	HMX451	Sara 3000 mast	1
2	HMX127	M10 x 20 allen bolt with flange	4
3	HMX1055	M6 washer	1
4	HMX564	M6 x 20 allen bolt	1
5	HMX572	M10 x 40 hex. bolt	1
6	HMX200	M10 locknut	1
7	HMX555	Linak actuator 230mm	1
8	X	See assembly lifting arm	1
9	HMX916	Cap D17	2
10	TMO.316	Sticker booklet	1
11	TMO.317	Sticker serial number	1
12	X	See assembly pushbar	1
13	X	See assembly electronics holder	1
14	HMX1099	Cable assembly Sara (part not illustrated)	1
15	HMX586	AMP plug male (pk 25) (part not illustrated)	4
16	HMX141	AMP 2p plug female (part not illustrated)	2
17	HMX977	Console Sara 3000 (part not illustrated)	1
18	TMO.324	Sticker US Sara (part not illustrated)	1
Items on page 15		Pushbar	
1	HMX878	Pushbar left	1
2	HMX877	Pushbar right	1
3	HMX232	Cap D28	2
4	HMX200	M10 locknut	2
5	HMX916	Cap D17 (grey)	4
6	HMX923	M10 x 65 hex. bolt	2
Items on page 16		Lifting arm	
1	HMX65	Lifting arm standard	1
or	HMX659	Lifting arm w/o horizontal grips	1
2	HMX570	Handgrip anti-skid	2
3	HMX185	Cap D22 flat	2
4	HMX916	Cap D17 (grey)	2
5	HMX200	M10 locknut	1
6	HMX320	Lock washer	1
7	HMX573	M10 x 55 hex.bolt	1
8	HMX605	M10 x 12 x 12 allen bolt	2
9	HMX868	Cap D25	2

B - Mast


Item No	Part Number	Description	Qty.
Items on page 17		Back plate	
1	HMX98	Backplate	1
2	HMX972	Lower plastic cap	1
3	HMX961	Upper plastic cap	1
4	HMX247	M5 Washer	5
5	HMX496	Poprivet 4.8 x 12	5
6	TMO.331	DC 50V / current 6A (part not illustrated)	1
Items on page 18		Console	
1	X	See composition electronics holder	X
2	X	See composition electronics holder	X
3	HMX1061	Cable assembly with plug	1
4	HMX978	Dual control switch	1
5	HMX979	Bezel	1
6	HMX998	Switch H/L	1
7	TMO.323	Sticker green / red arrow	1
8	HMX824	Plastic pin plug	1
9	HMX332	Nylon washer 4.3 x 18 x 2	2
10	HMX420	M4 x 10 cylindrical screw	2
11	HMX975	PCB	1
12	HMX922	M3 x 20 cylindrical screw	2
13	TMO.330	Sticker Sara 3000	1
14	HMX976	Speaker battery (part not illustrated)	1
15	HMX1062	Cable assembly blue/yellow wire plug (part not illustrated)	1
16	HMX1060	Cable assembly IEC bulgin plug (part not illustrated)	1
Items on page 19		Plastic cap assembly (electronics holder)	
1	HMX973	Plastic cap electronics	1
2	HMX974	Battery lifetime indication	1
3	HMX873	M2.5 nut	2
4	HMX1063	Cable electronics	1
5	HMX913	Label hours run	1
6	LABARJ	Label set Sara 3000 ARJO sticker	1
7	HMX1054	Distance bush 3.1 x 6 x 4	2
8	HMX875	M2.5 washer	2
9	HMX925	M2.5 x 16 cylindrical screw countersunk head	2
10	HMX981	Protection strip battery Sara	1

B - Mast

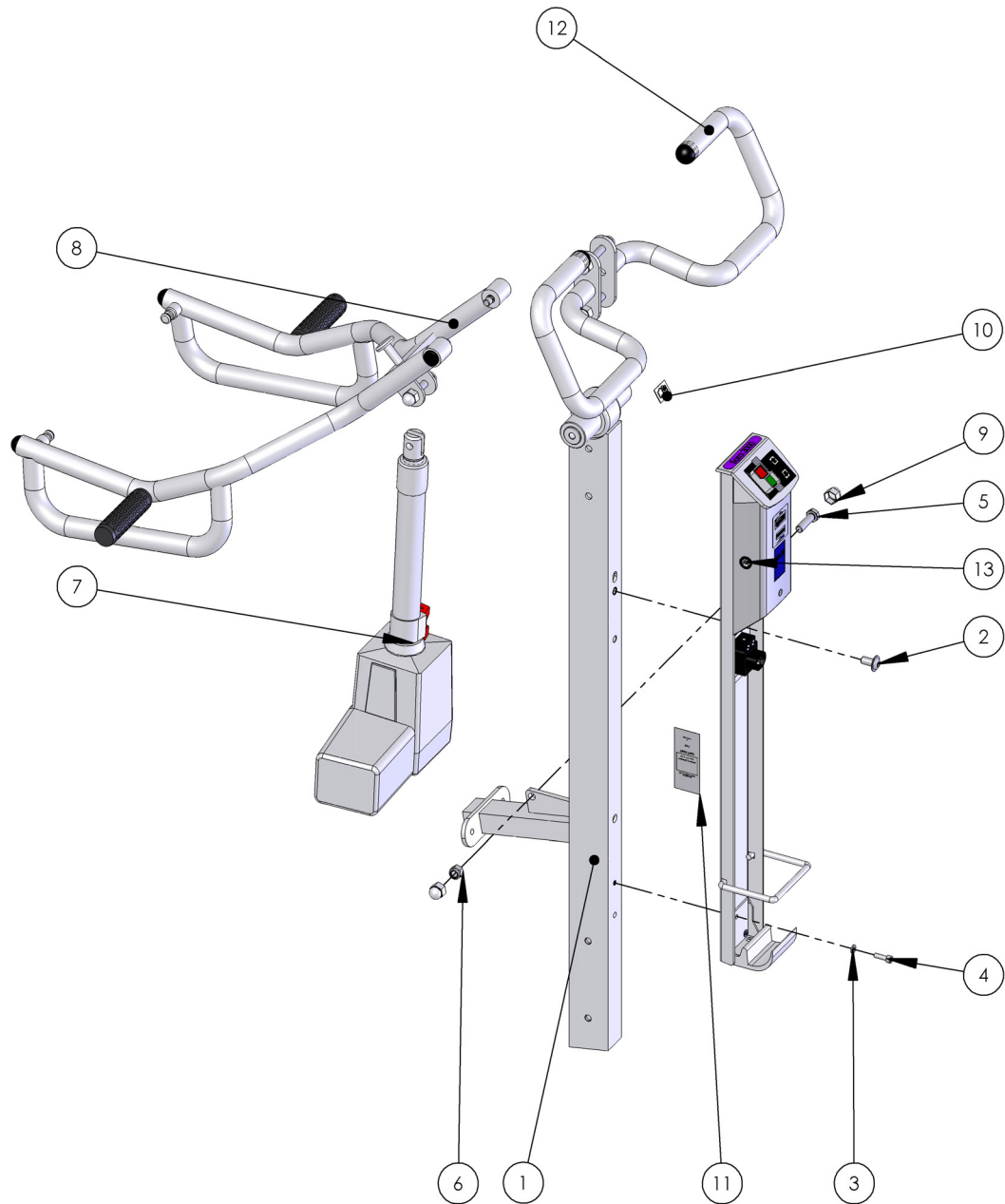


B - Mast

Push bar

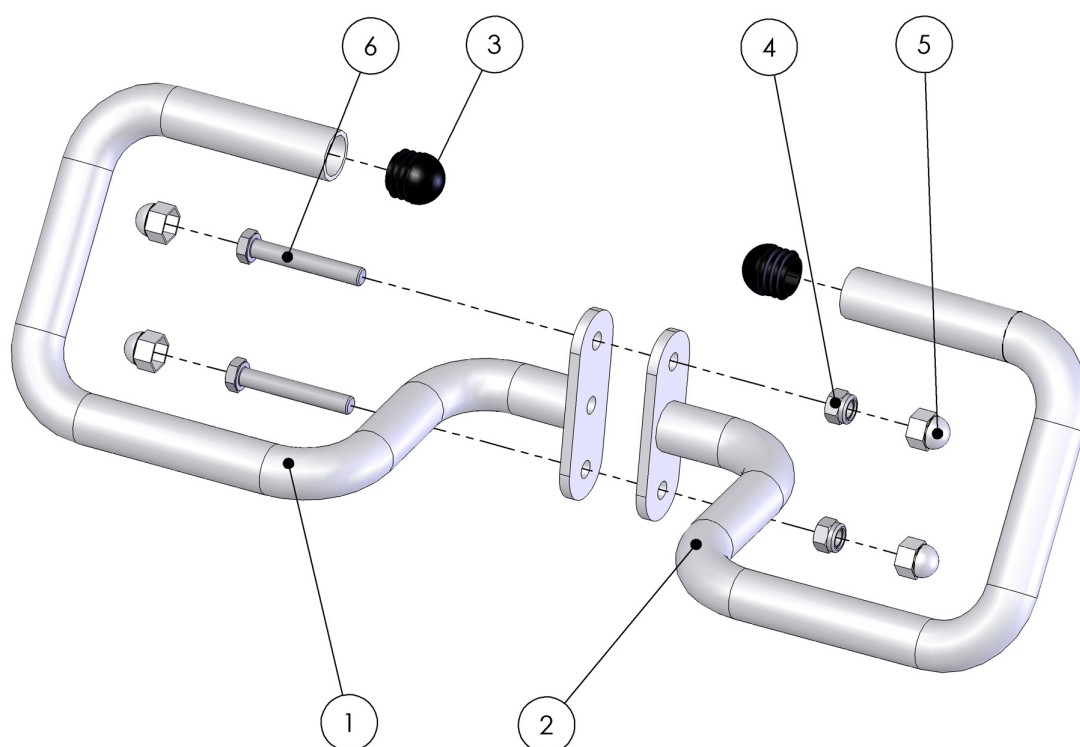


B - Mast

Lifting arm

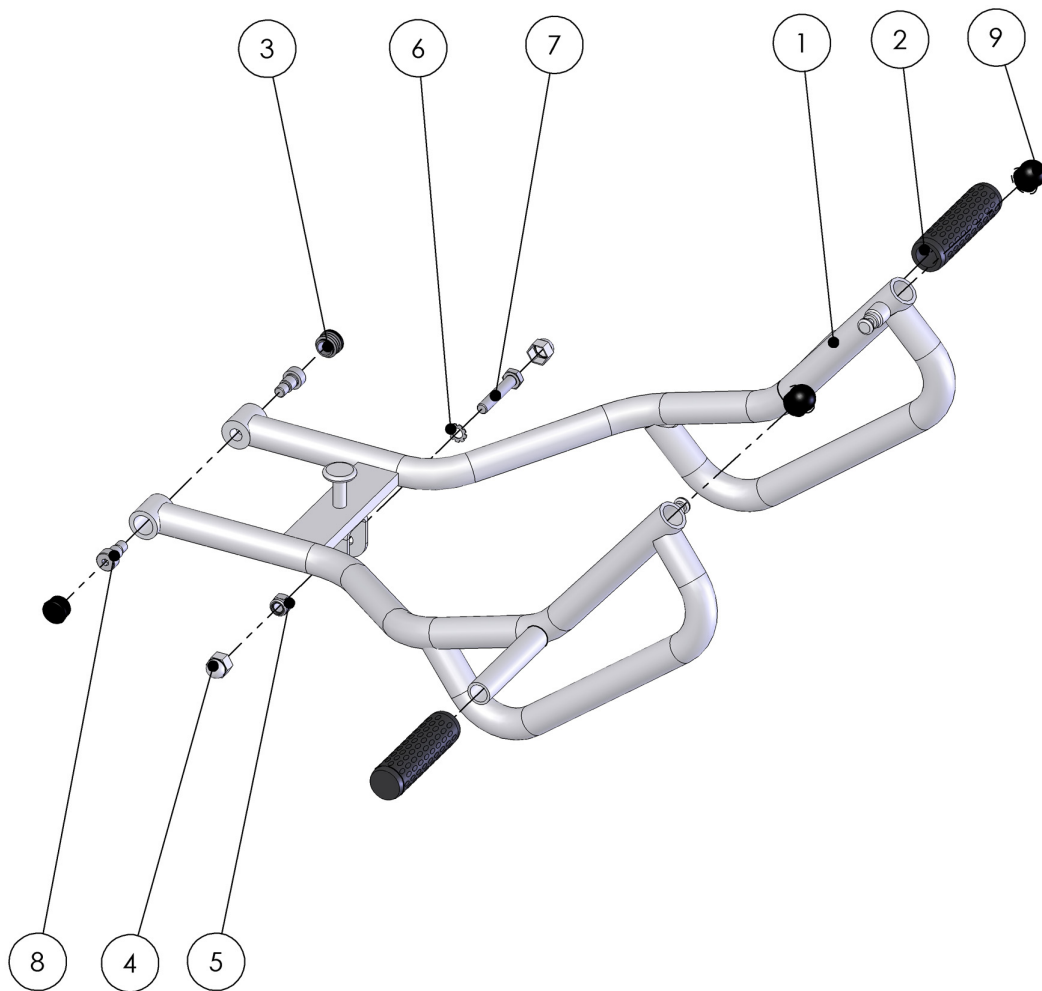


B - Mast

Back plate

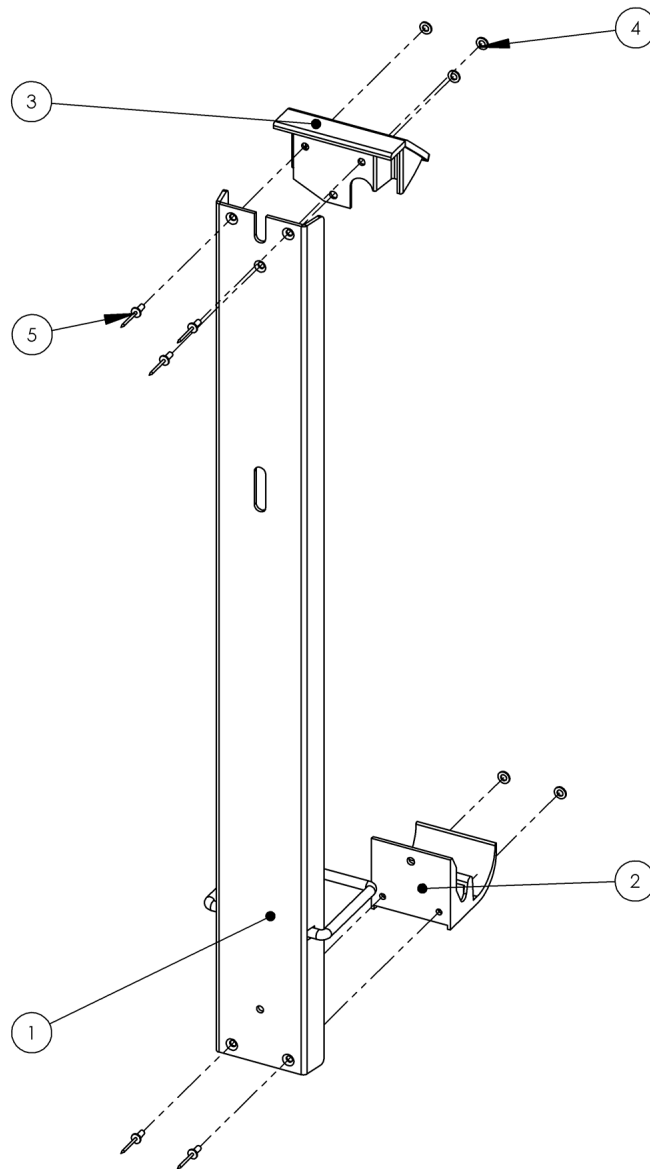


B - Mast

Console

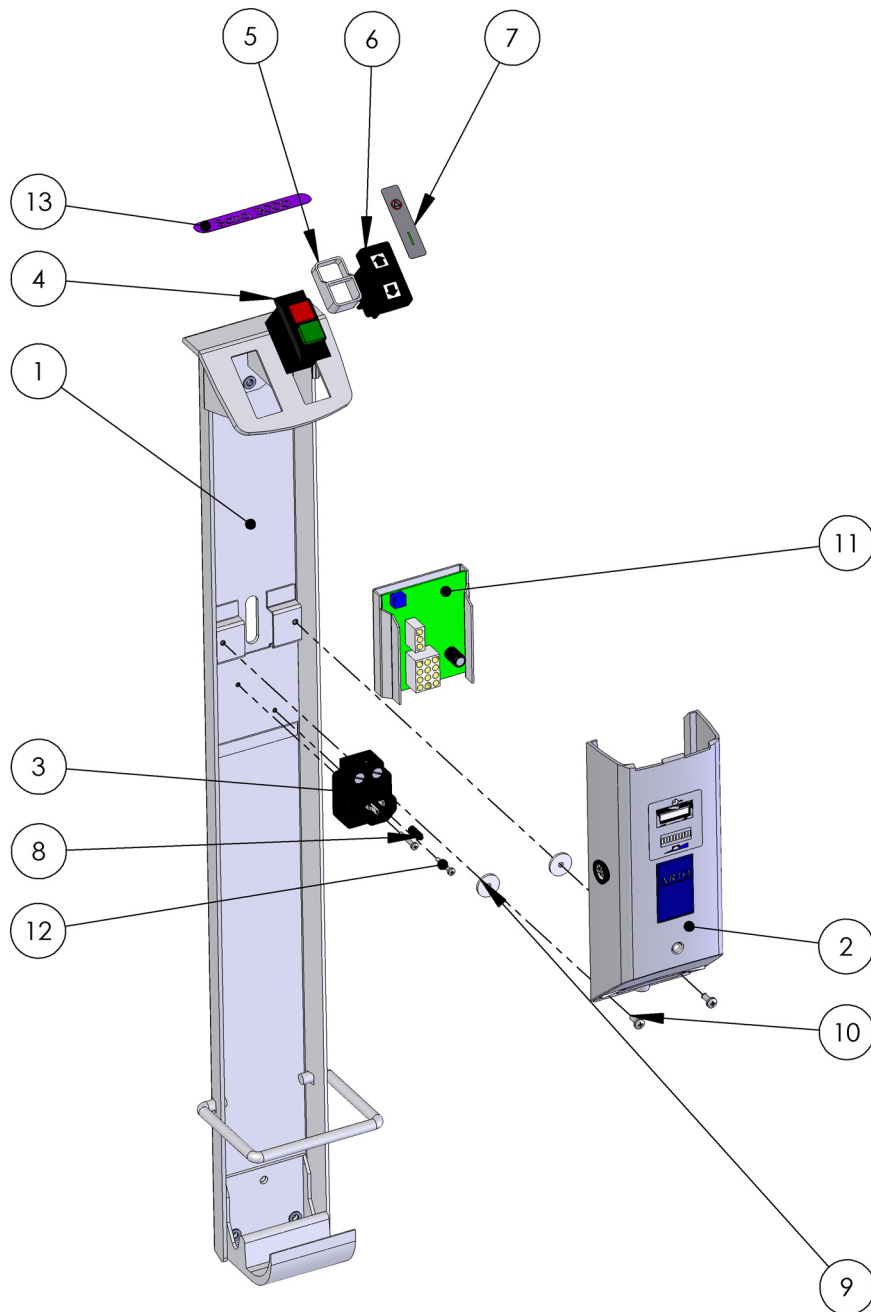

B - Mast

Plastic cap

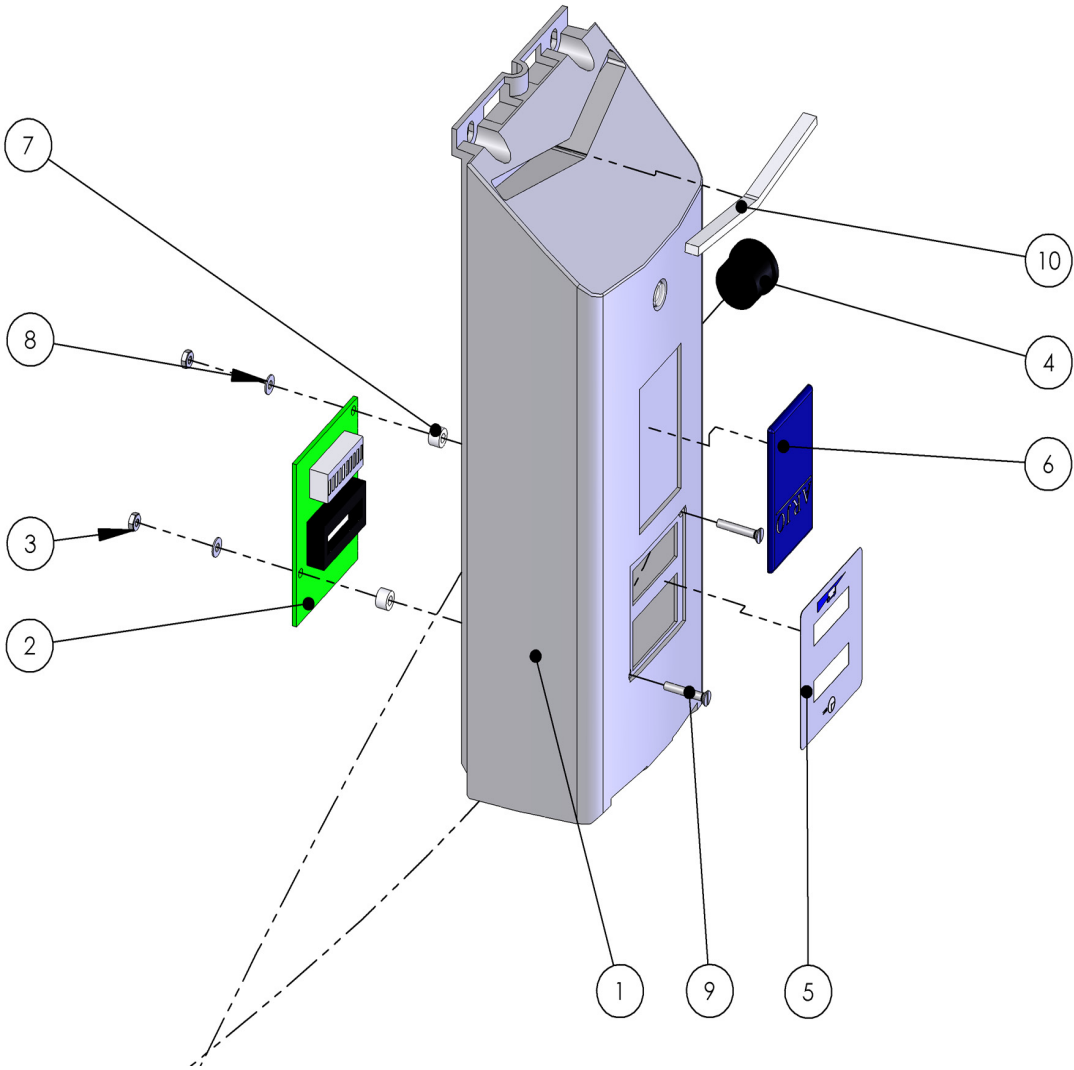

AUSTRALIA

ARJO Hospital Equipment Pty. Ltd.
205 Queensport Road, Murarrie,
Brisbane QLD 4172, Australia

PO Box 675, Bulimba
Brisbane QLD, 4171, Australia
Tel. (61) 7 3395 6311
Fax. (61) 7 3395 6712
Email: info@arjoaustralia.com.au

SVERIGE

ARJO INTERNATIONAL AB
Verstadsvägen 5
Box 61, 241 21 Eslov, Sweden
Tel. +46 413-645 00
Fax. +46 413-645 63
E-mail: arjo.international@arjo.com

DANMARK

ARJO SCANDINAVIA
Postboks 51, 4632 Bjæverskov
Tel. 45 93 27 37
Fax. 45 93 27 39
E-post: kundservice@arjo.se

NORGE

ARJO SCANDINAVIA
Ryenstubbene 2, 0679 Oslo
Tel. 98 28 11 70
Fax. 22 57 06 52
E-post: kundservice@arjo.se

BELGIQUE / BELGIË

ARJO N.V. Hospital Equipment NV / S.A.
Ternesselei 248
2160 WOMMELGEM
Tel. 03/353 91 00
Fax. 03/353 91 01
Email: info@arjo.be

CANADA

ARJO Canada Inc.
1575 South Gateway Road
Unit C, Mississauga, ONTARIO
L4W 5J1
Tel. - 800 665 4831
Fax.- 800 309 1116

ČESKÁ REPUBLIKA

ARJO Hospital Equipment s.r.o.
Hlinky 118
CZ-603 00 BRNO
Tel. 420 549 254 252
Fax. 420 541 213 550

DEUTSCHLAND

ARJO GmbH
Christof-Ruthof-Weg 6
55252 Mainz-Kastel, Germany
Tel. +49 (0)61 34-186 0
Fax. +49 (0)61 34-186 160
Email: info@arjo.de

ESPAÑA

ARJO SPAIN, S.A.
C/ San Rafael, nº 6
28108 ALCOBENDAS (MADRID)
SPAIN
Tel. +34 902 300 126
Fax. +34 902 300 191
Email: arjospain@arjospain.es

FRANCE

ARJO Equipements Hospitaliers S.A.
45, Avenue de l'europe
Eurocit BP133
F-59436 RONCQ CEDEX
FRANCE
Tel. 03 20 28 13 13
Fax. 03 20 28 13 14
Email: info@arjo.fr

GREAT BRITAIN

ARJO MED AB Ltd.
St. Catherine Street
Gloucester, GL1 2SL
ENGLAND U.K.
Tel. (08702) 430430
Fax. (01452) 428344

IRELAND

ARJO Ireland Ltd, EA House,
Damastown Industrial Estate,
Mulhuddart, Dublin 15, Eire
Tel. 00 35 31 8098960
Fax. 00 35 31 8098971

HONG KONG

ARJO Far East Ltd.
1001-1003 APEC Plaza,
49 Hoi Yuen Road, Kwun Tong,
Kowloon, HONG KONG
Tel. 852 2908 9553
Fax. 852 2508 1416

ITALIA

ARJO Italia SPA
Via Tor Vergata,
432 - 00133 ROMA
Tel. 06 87426211
Fax. 06 87426222
Email: promo@arjo.it

NEDERLAND

ARJO NEDERLAND B.V.
De Blomboogerd 8,
4003 BX TIEL,
Postbus 6116,
4000 HC TIEL
Tel. +31 (0)344 64 08 79
Fax. +31 (0)344 64 08 85
Email: info@arjo.nl

ÖSTERREICH

ARJO GmbH, Föhrenweg 5,
A-6065 THAUR, Austria
Tel. +43 (0)5223 493350
Fax. +43 (0)5223 493350 75
Email: office@arjo.at

POLAND

Getinge POLAND Sp. z o.o.
02-387 Warszawa
ul. Lirowa 27
Poland
Tel. +48 22 882-06-26..28
Fax +48 22 882-24-52
Email: wysokinski@getinge.pl

SCHWEIZ / SVIZZERA

ARJO INTERNATIONAL AG
Florenzstr. 1d, Postfach,
CH-4023 BASEL, Switzerland
Tel. +41 (0)61 337 9797.
Fax. +41 (0)61 331 4780
Email: arjo.international@arjo.com

SCHWEIZ

ARJO-SIC AG
Florenzstr. 1d, Postfach
4023 BASEL
Tel. +41 (0)61 337 97 77
Fax. +41 (0)61 311 97 42
Email: arjo.sic@arjo.ch

USA

ARJO, Inc.
50 North Gary Avenue
Roselle, IL 60172
Tel. 1-800-323-1245
Fax. 1-888-594-ARJO(2756)



www.arjo.com



© Medibo Medical Products NV
Heikant 5, 3930 Hamont Achel
Belgium

If your country is not listed here,
please contact your local distribu-
tor, or ARJO International AB, Box
61, S-241 21 ESLÖV, SWE-
DEN.
Tel +46 413 64500.
Fax +46 413 555586.

GETINGE GROUP is a leading global provider of equipment and systems that con-
tribute to quality enhancement and cost efficiency within healthcare and life sci-
ences, equipment, services, and technologies are supplied under the brands
ARJO for resident hygiene, resident handling and wound care, GETINGE for infec-
tion control and prevention within healthcare and life science and MAQUET for sur-
gical workplaces, cardiopulmonary and critical care.